

NITRILE GLOVES

Medical Examination Gloves

- ✓ The safe and practical choice for every task
- ✓ Cuddle your hands protect to your health




Gloves are the important tool in **preventing HAIs** (Healthcare-associated Infection) & **cross-contamination**.

Nitrile gloves made from **Nitrile Butadiene Rubber (NBR)**.

That made gloves are widely used due to their **excellent protection**.



✓ Powder free

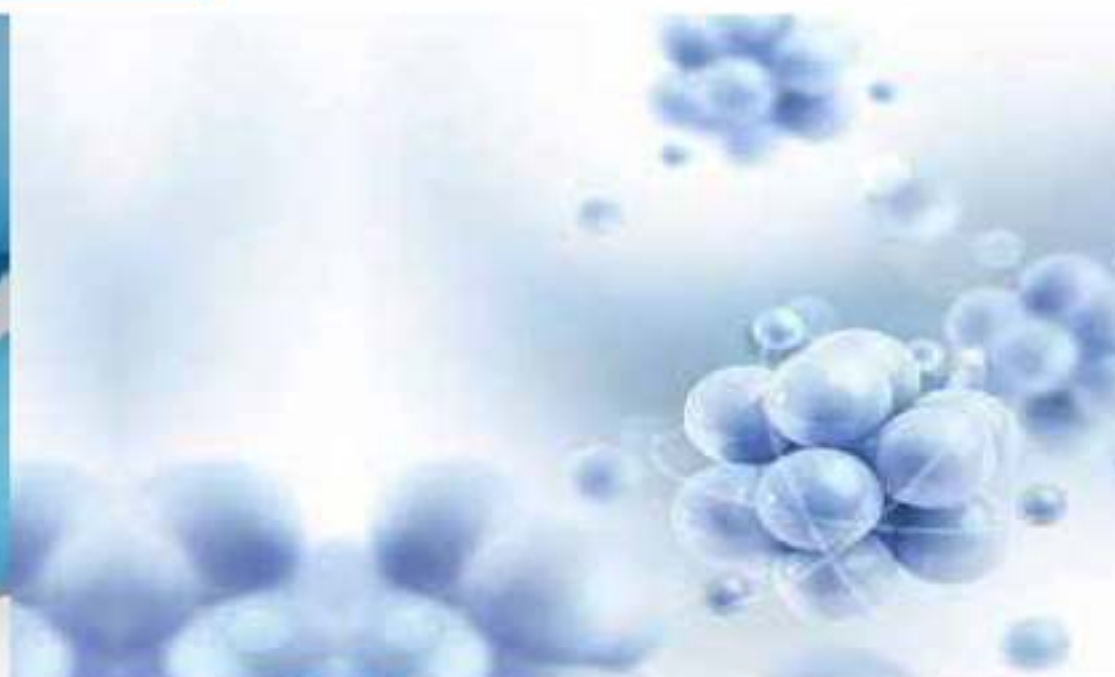

✓ Non-leaching antimicrobial



✓ Ambidextrous

Good Quality Price

The excellent range includes:



Excellent Protect




Thicker and stronger gloves



Elasticity with 3000 PSI






Chemical resistance

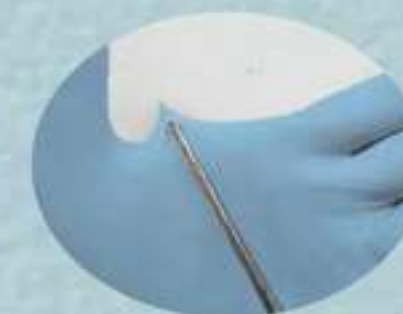

SIZE

How to choose the right gloves?

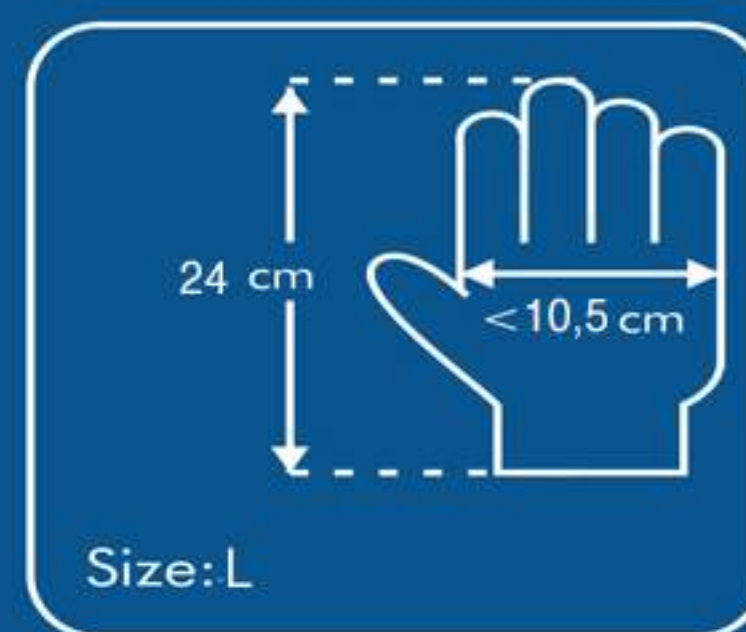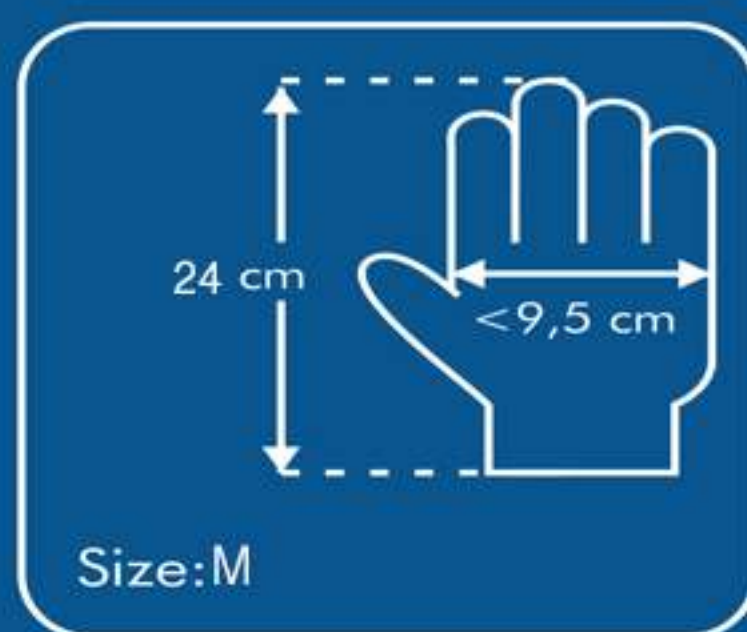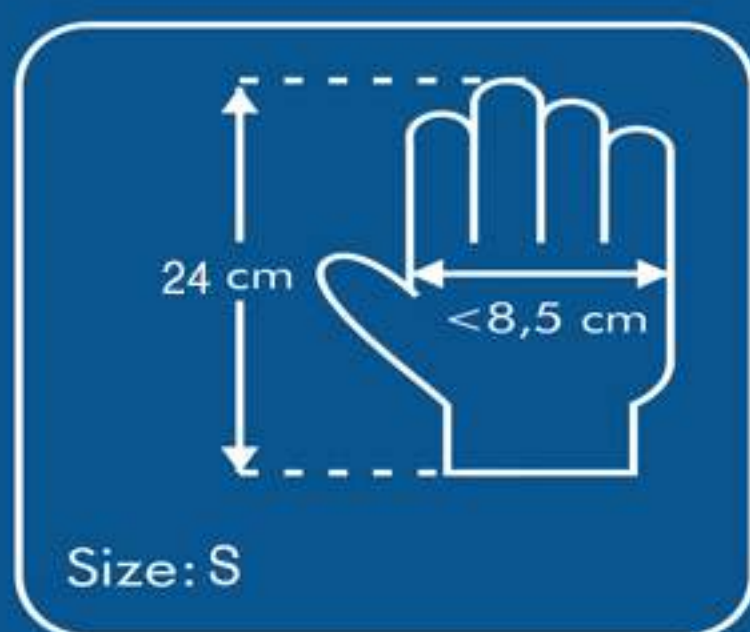

INFORMATION


Name	Nam Viet Nitrile Gloves	Package	100pcs/box
Type	Medical Examination Gloves	Weight	400 gram to 650 gram
Size	S/M/L/XL	Carton	10 boxes/Carton
Material	Nitrile Butadiene Rubber (NBR)	Gross Weight	4,5Kg
Color	White/Blue	20ft Container	1700 Cartons
Origin	Vietnam	40ft Container	3000 Cartons

Agreed Technical Data & Specification of Gloves Manufacturer

NAM VIET GLOVES JOINT STOCK COMPANY

To be stamped by Seller

Tên khách hàng	NAM VIET	Bảng màu					Kích thước	345 x 228 x 212 mm	Tỷ lệ	100 %
Tên sản phẩm	Thùng NVIMEDIC Nitrile Blue Final_2017	1	2	3	4	5	Mã số bản	PLE00003	Thiết kế	Mr.Nhan
Ngày nhận đơn	20-06-2018						Mã số khuôn		Lưu trữ	POLIFE \20-06-2018\Final_2017

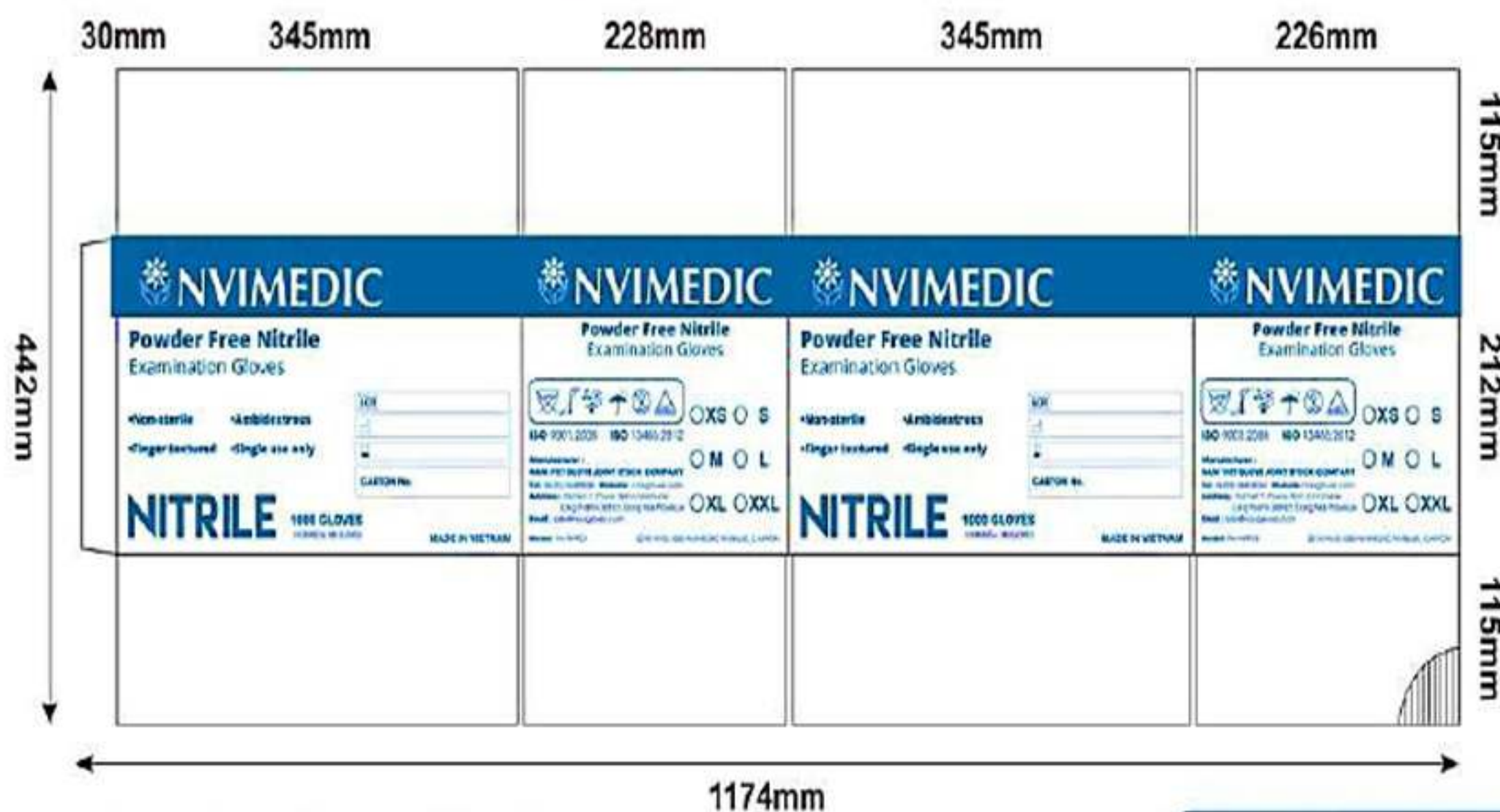

Powder Free Nitrile
Examination Gloves

- Non-sterile
- Ambidextrous
- Finger textured
- Single use only

NITRILE 1000 GLOVES
(10 BOXES x 100 GLOVES)

MADE IN VIETNAM



ISO 9001:2008 ISO 13485:2012

Manufacturer :
NAM VIET GLOVE JOINT STOCK COMPANY
Tel: (8428) 38408038 Website: navigloves.com
Address : Hamlet 7, Phuoc Binh Commune
Long Thanh District, Dong Nai Province
Email : sale@navigloves.com

Model: NV-NFFEX

2018 NVG, 000-NVIMEDIC NVBLUE_CARTON

OX S

M L

XL XXL

P.Kinh doanh duyệt mẫu

Khách hàng duyệt mẫu

Exhibit B:
Box design:

- 1 HỘP 100 CÁI – 10 HỘP/THÙNG

Tên sản phẩm: _____

2016_NVG_003-NVIMEDIC NVIBLUE_BOX_STICK SIZE





Food and Drug Administration
10903 New Hampshire Avenue
Document Control Center - WO66-G609
Silver Spring, MD 20993-0002

November 7, 2016

Nam Viet Glove Joint Stock Company
Ooi Loon Seng
QA Manager
Hamlet 7, Phuoc Binh Commune
Long Thanh District
Ho Chi Minh
Dong Nai Province, VN 815300

Re: K162113

Trade/Device Name: Powder Free Nitrile Examination Gloves (Blue)
Regulation Number: 21 CFR 880.6250
Regulation Name: Patient Examination Glove
Regulatory Class: Class I
Product Code: LZA
Dated: September 26, 2016
Received: October 3, 2016

Dear Ooi Loon Seng:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803); good manufacturing practice requirements as set forth in

the quality systems (QS) regulation (21 CFR Part 820); and if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

If you desire specific advice for your device on our labeling regulation (21 CFR Part 801), please contact the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>. Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <http://www.fda.gov/MedicalDevices/Safety/ReportaProblem/default.htm> for the CDRH's Office of Surveillance and Biometrics/Division of Postmarket Surveillance.

You may obtain other general information on your responsibilities under the Act from the Division of Industry and Consumer Education at its toll-free number (800) 638-2041 or (301) 796-7100 or at its Internet address

<http://www.fda.gov/MedicalDevices/ResourcesforYou/Industry/default.htm>.

Sincerely,

Michael J. Ryan -S

for Tina Kiang, Ph.D.
Acting Director
Division of Anesthesiology,
General Hospital, Respiratory,
Infection Control, and Dental Devices
Office of Device Evaluation
Center for Devices and Radiological Health

Enclosure

DEPARTMENT OF HEALTH AND HUMAN SERVICES
Food and Drug Administration

Form Approved: OMB No. 0910-0120
Expiration Date: January 31, 2017
See PRA Statement below.

Indications for Use

510(k) Number (if known)

K162113

Device Name

Powder Free Nitrile Examination Gloves (Blue)

Indications for Use (Describe)

A patient examination glove is a disposable device intended for medical purposes that is worn on the examiner's hands or fingers to prevent contamination between patient and examiner.

Type of Use (Select one or both, as applicable)

☐ Prescription Use (Part 21 CFR 801 Subpart D)

☒ Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASTaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

510(k) SUMMARY
510(k) Number: K162113

1.0 Submitter :

Nam Viet Glove Joint Stock Company,
Hamlet 7, Phuoc Binh Commune,
Long Thanh District 815300,
Dong Nai Province,
Vietnam.

Phone No. : +84 08 3514 7788
Fax No. : +84 08 3840 8506
Contact Person : Ooi Loon Seng (Mdm)
Email : lssooi@navigloves.com

Date of Preparation : 01 November 2016

2.0 Name of the Device

Powder Free Nitrile Examination Gloves (Blue)

Common Name : Nitrile Patient Exam Gloves
Classification Name : Patient Examination Gloves (21 CFR 880.6250)
Product Code : Nitrile – LZA

3.0 Identification of The Legally Marketed Devices That Equivalency is Claimed:

Predicate Device :

RS Safe Blue Nitrile Medical Examination Gloves Powder Free (Non-Sterile)
Company : Riverstone Resources Sdn. Bhd.
510(K) : K100603
Regulatory Class I
Product Code : LZA

4.0 Description of the Device:

The Powder Free Nitrile Examination Gloves (Blue) meets all the requirements of ASTM Specification D6319-10 - Standard Specification for Nitrile Examination Gloves for Medical Application.

5.0 Intended Use of the Device

The Powder Free Nitrile Examination Glove (Blue) is a disposable device intended for medical purposes that is worn on the examiner's hands or fingers to prevent contamination between patient and examiner.

6.0 Summary of the Technological Characteristics of the Device:

The Powder Free Nitrile Examination Gloves (Blue) are summarized with the following technological characteristics and is substantially equivalent to the predicate device with regard to

Traditional 510(k) Premarket Notification
Powder Free Nitrile Examination Gloves (Blue)

physical characteristics, design, product features, and intended use. Both gloves are made with nitrile latex and meets ASTM Specification D6319-10 - Standard Specification for Nitrile Examination Gloves for Medical Application or equivalent standards.

CHARACTERISTICS	STANDARDS & ACCEPTANCE CRITERIA	DEVICE PERFORMANCE
Dimensions	ASTM D6319-10: Length: Size XS & S – min. 220mm Size M & L – min. 230mm	Meets ASTM D6319-10 Standard requirements Minimal Value for Length: 240mm
	Width – min. 95 ± 10mm	Minimal Value for Width: 93mm
Physical Properties	ASTM D6319-10: Tensile Strength: Before Aging: 14 MPa After Aging: 14 MPa Ultimate Elongation: Before Aging: min. 500% After Aging: min. 400%	Meets ASTM D6319-10 Standard requirements Tensile Strength: Before Aging: Minimal Value: 16.20 MPa After Accelerated Aging: Minimal Value: 16.16 MPa Ultimate Elongation: Before Aging Minimal Value: 661% After Aging Minimal Value: 630%
Thickness	ASTM D6319-10: Min. 0.05mm	Meets ASTM D6319-10 Standard requirements Palm – Minimal Value: 0.06mm Finger – Minimal Value: 0.13mm
Biocompatibility	ISO 10993-10: 2010 Biological evaluation of medical devices – Part 10: Tests for irritation and skin sensitization	Pass (Under the conditions of the study, the test article is a non-irritant)
	ISO 10993-10: 2010 Biological evaluation of medical devices – Part 10: Tests for irritation and skin sensitization	Pass (Under the conditions of the study, the test article is a non-sensitizer)
Freedom from Pinholes	21 CFR 800.20; ASTM D5151-06 AQL 2.5	Meets 21 CFR 800.20 and ASTM D5151-06 Standard requirements Passes AQL 2.5
Powder Residual	ASTM D6124-06: ≤2 mg / glove	Meets ASTM D6124-06 Standard requirements Minimal Value: 0.68 mg

7.0 Substantial Equivalent Based on Assessment of Non-Clinical Performance Data

The performance test data of non-clinical tests that support a determination of substantial equivalence is the same as described in section 6.0 above and meet ASTM standards, and FDA requirements for water leak test on pinhole AQL.

8.0 Substantial Equivalent Based on Assessment of Clinical Performance Data

Traditional 510(k) Premarket Notification
Powder Free Nitrile Examination Gloves (Blue)

Not applicable - Clinical data is not needed for gloves or most devices cleared by the 510(k) process.

9.0 Substantial Equivalence Comparison

The Powder Free Nitrile Examination Gloves (Blue) is substantially equivalent to the predicate device with respect to intended use, product features and the technological characteristics. The substantial equivalence comparison is presented in the table below:

Characteristic and parameters	Riverstone Resources Sdn. Bhd. K100603 RS Safe Blue Nitrile Medical Examination Gloves Powder Free (Non-Sterile) (Predicate)	Powder Free Nitrile Examination Gloves (Blue) (Proposed) – K162113	Substantial Equivalence (SE)
Product Code	LZA	LZA	Same
Intended use	Intended for medical purposes that is worn on the examiner's hands or fingers to prevent contamination between patient and examiner.	Intended for medical purposes that is worn on the examiner's hands or fingers to prevent contamination between patient and examiner.	Same
Indications for Use Statement	A powder free patient examination gloves is a disposable device intended for medical purposes that is worn on the examiner's hand or finger to prevent contamination between patient and examiner.	A patient examination glove is a disposable device intended for medical purposes that is worn on the examiner's hand or finger to prevent contamination between patient and examiner.	Same
Device Material	Nitrile Synthetic Latex	Nitrile Synthetic Latex	Same
Color	Blue	Blue	Same
Additives	No flavor additive	No flavor additive	Same
Instruction For Use on Labeling	Single Use Only	Single Use Only	Same
Construction	Ambidextrous	Ambidextrous	Same
Sterility	Non-sterile	Non-sterile	Same
Acceptance Criteria	ASTM D6319-10	ASTM D6319-10	Same
Device Tolerances & Specifications :			
Dimensions	Meets ASTM D6319-10 Overall Length: Size XS & S – min. 220mm Size M & L – min. 230mm Width (± 10mm) Size XS – 70mm Size S – 80mm Size M – 95mm Size L – 110mm Size XL – 120mm Thickness at Finger – min. 0.05mm	Meets ASTM D6319-10 Overall Length: Size XS & S – min. 220mm Size M & L – min. 230mm Width (± 10mm) Size XS – 70mm Size S – 80mm Size M – 95mm Size L – 110mm Size XL – 120mm Thickness at Finger – min. 0.05mm	Same

Traditional 510(k) Premarket Notification
Powder Free Nitrile Examination Gloves (Blue)

	Thickness at Palm – min. 0.05mm	Thickness at Palm – min. 0.05mm	
Physical Properties	Meets ASTM D6319-10 : <u>Before Aging</u> Tensile Strength – min. 14.0 MPa Ultimate Elongation – min. 500% <u>After Aging</u> Tensile Strength – min. 14.0 MPa Ultimate Elongation – min. 400%	Meets ASTM D6319-10 : <u>Before Aging</u> Tensile Strength – min. 14.0 MPa Ultimate Elongation – min. 500% <u>After Aging</u> Tensile Strength – min. 14.0 MPa Ultimate Elongation – min. 400%	
Biocompatibility Test a. Irritation Tests b. Skin Sensitization Tests	Under the conditions of the study, the test article is a non-irritant and non-sensitizer.	Under the conditions of the study, the test article is a non-irritant and non-sensitizer.	Same
Residual Powder test	Meets ASTM D6124-06: ≤2 mg / glove	Meets ASTM D6124-06	Same
Freedom from Holes	Meets ASTM D5151-06 AQL 2.5	Meets ASTM D5151-06 AQL 2.5	Same

10.0 Conclusion

The Powder Free Nitrile Examination Gloves (Blue) meet all of the requirements of FDA-recognized consensus standards; ASTM D6319-10, ASTM D5151-06, ASTM D6124-06 and ISO 10993-10: 2010 and meet our labeling claims and pinhole acceptable quality level (AQL) as shown above.

There are no significant differences between the two products and are identical in terms of intended use, materials and performance.

The conclusion drawn from the nonclinical tests demonstrate that the device is as safe, as effective, and performs as well as or better than the predicate device.

CÔNG TY CP TM – DV – XNK
VIÊN PHÁT

CỘNG HÒA XÃ HỘI CHỦ NGHĨA VIỆT NAM
Độc lập - Tự do - Hạnh phúc

Số: 180801-VP/180000019/PCBPL-
BYT

TP.HCM, ngày 01 tháng 08 năm 2018

BẢN PHÂN LOẠI TRANG THIẾT BỊ Y TẾ

Kính gửi: Công ty Cổ phần Găng tay Nam Việt

Căn cứ Nghị định số 36/2016/NĐ-CP ngày 15 tháng 5 năm 2016 của Chính phủ về quản lý trang thiết bị y tế;

Căn cứ Thông tư số 39/2016/TT-BYT ngày 28 tháng 10 năm 2016 của Bộ Y tế, quy định chi tiết việc phân loại trang thiết bị y tế;

Căn cứ Thông tư số 42/2016/TT-BYT ngày 15 tháng 11 năm 2016 của Bộ Y tế, quy định thừa nhận kết quả phân loại trang thiết bị y tế;

Nguyên tắc được sử dụng để phân loại:

Quy tắc 5 – Trang thiết bị y tế không phải trang thiết bị y tế chẩn đoán in vitro

Chúng tôi phân loại trang thiết bị y tế như sau:

STT	Tên Trang thiết bị Y Tế	Chủng loại/ Mã sản phẩm	Hãng, nước sản xuất	Hãng, nước chủ sở hữu	Loại trang thiết bị y tế	Quy tắc sử dụng
1	Găng tay khám bệnh không bột (loại cao su tổng hợp) Powder free Nitrile examination gloves	NV-NPFEX / XS,S,M,L,XL &XXL	Công ty Cổ Phần Găng Tay Nam Việt - Việt Nam	Công ty Cổ Phần Găng Tay Nam Việt - Việt Nam	A	5
2	Găng tay khám bệnh không bột (loại cao su thiên nhiên) Powder free Latex examination gloves	NV-LPFEX / XS,S,M,L,XL &XXL	Công ty Cổ Phần Găng Tay Nam Việt - Việt Nam	Công ty Cổ Phần Găng Tay Nam Việt - Việt Nam	A	5

Giám đốc



HA NGOC MINH PHONG

Số: 180000006/PCBSX-ĐN

Tỉnh Đồng Nai, ngày 11 tháng 09 năm 2018

PHIẾU TIẾP NHẬN**Hồ sơ công bố đủ điều kiện sản xuất trang thiết bị y tế**

1. Tên cơ sở công bố: CÔNG TY CỔ PHẦN GĂNG TAY NAM VIỆT
2. Địa chỉ: 23A NGUYỄN HỮU CẢNH, Phường 22, Quận Bình Thạnh, Thành phố Hồ Chí Minh
(Sản xuất tại: CÔNG TY CỔ PHẦN GĂNG TAY NAM VIỆT; Địa chỉ: ẤP 07, Xã Phước Bình, Huyện Long Thành, Tỉnh Đồng Nai)
3. Điện thoại: 02835147788 Fax:
4. Số văn bản đề nghị của cơ sở: 240701/2018-VB Ngày: 29/08/2018
5. Tên trang thiết bị y tế cơ sở công bố sản xuất:
GĂNG TAY KHÁM BỆNH KHÔNG BỘT (LOẠI CAO SU TỔNG HỢP) POWDER FREE
NITRILE EXAMINATION GLOVES , GĂNG TAY KHÁM BỆNH KHÔNG BỘT (LOẠI CAO
SU THIÊN NHIÊN) POWDER FREE LATEX EXAMINATION GLOVES
6. Thành phần hồ sơ:

1	Văn bản phân công, bổ nhiệm người phụ trách chuyên môn của cơ sở sản xuất	x
2	Bản xác nhận thời gian công tác	x
3	Văn bằng, chứng chỉ đào tạo về kỹ thuật thiết bị y tế hoặc quản lý thiết bị y tế của người phụ trách chuyên môn	x
4	Văn bản công bố đủ điều kiện sản xuất	x
5	Bản kê khai nhân sự	x
6	Văn bản phân công, bổ nhiệm người phụ trách chuyên môn của cơ sở sản xuất	x
7	Giấy chứng nhận đạt tiêu chuẩn quản lý chất lượng	x
8	Hồ sơ chứng minh địa điểm, diện tích, nhà xưởng sản xuất phù hợp với yêu cầu của loại trang thiết bị y tế mà cơ sở sản xuất	x
9	Hồ sơ về thiết bị và quy trình sản xuất, kiểm tra chất lượng phù hợp với yêu cầu của loại trang thiết bị y tế mà cơ sở sản xuất	x
10	Hợp đồng với cơ sở đủ năng lực kiểm tra chất lượng để kiểm tra chất lượng trang thiết bị y tế mà cơ sở sản xuất	x
11	Hồ sơ về kho tàng bảo quản trang thiết bị y tế	x
12	Hồ sơ về phương tiện vận chuyển trang thiết bị y tế	x

NGƯỜI TIẾP NHẬN HỒ SƠ

Phan Huy Anh Vũ



PRODUCTS SPECIFICATION THÔNG SỐ SẢN PHẨM

1. Product Number and Description/ Số sản phẩm và mô tả :

Product Code <i>Mã sản phẩm</i>	Product Description <i>Mô tả sản phẩm</i>	Design & Features <i>Kiểu dáng và đặc trưng</i>	Product Color <i>Màu sản phẩm</i>	Classification <i>Phân loại</i>
LPF-FT50-24Z	Non Sterile Chlorinated Powder Free Latex Examination Glove (M= 5.0gm +/- 0.2gm) <i>Găng tay cao su y tế không bột (M= 5.0g +/- 0.2g)</i>	Finger Textured <i>Vân nhám đầu ngón tay</i> Beaded Cuff <i>Cuộn mép ở cổ tay</i> Ambidextrous <i>Không phân biệt mặt</i>	Natural <i>Tự nhiên</i>	Class 1 <i>Cấp 1</i>
Material <i>Nguyên liệu</i>	Natural Rubber <i>Cao su tự nhiên</i>			

2. Physical Dimension/ Kích thước vật lý:

2.1 Width and Length/ Chiều rộng và chiều dài:

Size <i>Kích cỡ</i>	Total Length (mm) <i>Chiều dài tổng (mm)</i>	Width (mm) <i>Chiều rộng (mm)</i>	Weight (gm) Weight (gm) <i>Khối lượng (g)</i>
X-Small <i>Cực nhỏ</i>	Min 240 <i>Tối thiểu 240</i>	75 +/- 5	4.0 +/- 0.2
Small <i>Nhỏ</i>	Min 240 <i>Tối thiểu 240</i>	85 +/- 5	4.5 +/- 0.2
Medium <i>Trung bình</i>	Min 240 <i>Tối thiểu 240</i>	95 +/- 5	5.0 +/- 0.2
Large <i>Lớn</i>	Min 240 <i>Tối thiểu 240</i>	106 +/- 5	5.5 +/- 0.2
X-Large <i>Cực lớn</i>	Min 240 <i>Tối thiểu 240</i>	115 +/- 5	6.0 +/- 0.2
Extra Extra-Large <i>Siêu lớn</i>	Min 240 <i>Tối thiểu 240</i>	>120	6.5 +/- 0.2

2.2 Thickness (Single Wall)/ Độ dày (một lớp):

Location of Thickness Measurement <i>Vị trí đo độ dày</i>	Single Wall Thickness (mm) <i>Chiều dày một lớp (mm)</i>	Reference Standard <i>Tiêu chuẩn tham khảo</i>
Palm (at center of palm) <i>Lòng bàn tay (tại điểm giữa của lòng bàn tay)</i>	Min 0.08	ASTM D 3578
Fingertip (at 13mm from the extreme tip) <i>Đầu ngón tay (cách 13mm tính từ đỉnh ngón giữa)</i>	Min 0.08	ASTM D 3578

3. Physical Properties/ Tính chất vật lý:

Characteristics <i>Đặc tính</i>	Reference Standard <i>Tiêu chuẩn tham khảo</i>	Before Ageing <i>Trước lão hóa</i>	After Ageing 7 days @ 70°C ± 2 hours Sau lão hóa 7 ngày (ở 70°C) ± 2 giờ
Force at Break (N) <i>Lực căng tối đa (N)</i>	En 455, Part 2	Min 6 (N)	Min 6 (N)
Tensile Strength (MPa) <i>Sức căng kéo dài (MPa)</i>	ASTM D 3578	Min 18 Mpa	Min 14 Mpa
Elongation at Break (%) <i>Độ dẻo dài tối đa (%)</i>	ASTM D 3578	Min 650%	Min 500%



PRODUCTS SPECIFICATION THÔNG SỐ SẢN PHẨM

4. Quality Assurance/ Đảm bảo chất lượng:

Characteristics <i>Đặc tính</i>	Inspection Level & Sampling Plan <i>Mức độ kiểm tra & và kế hoạch lấy mẫu</i>	AQL & Acceptance Limit <i>AQL & Giới hạn chấp nhận</i>
Pinhole (Reference EN 455-1) <i>Lỗ rò (tham khảo EN 455-1)</i>	G-1 (Single Sampling)	AQL 1.5
Visual Major Defects <i>Lỗi trực quan chính</i>	G-1 (Single Sampling)	AQL 2.5
Visual Minor Defects <i>Lỗi trực quan nhỏ</i>	G-1 (Single Sampling)	AQL 4.0
Physical Dimension <i>Kích thước vật lý</i>	S-2	AQL 4.0
Powder Residual Limit <i>Giới hạn dư lượng bột</i>	As per ASTM D6124-06 Test Method <i>Phương pháp kiểm tra theo như ASTM D6124-06</i>	Below 2mg/glove <i>Dưới 2mg/ găng</i>
Water Extractable Protein	As per ASTM D5712-15 Test Method <i>Phương pháp kiểm tra theo như ASTM D5712-15</i>	Below 50µg/ dm ² <i>Dưới 50µg/ dm²</i>

5. Shelf Life/ Thời gian sống:

Under normal storage condition, gloves shall have a shelf life of minimum 3 years from date of manufacture.
Dưới điều kiện kho thông thường, găng tay sẽ có thời gian sống tối thiểu 3 năm kể từ ngày sản xuất.

6. Suggested Storage Condition/ Điều kiện kho được đề nghị:

Storage Condition <i>Điều kiện kho</i>	
Temperature range <i>Phạm vi nhiệt độ</i>	Recommended storage temperature shall not exceed 40 degrees Celsius. Recommended temperature is between 20 to 25 degrees Celsius. <i>Nhiệt độ kho được đề nghị sẽ không vượt quá 40°C. Nhiệt độ tối ưu được đề nghị nằm trong khoảng 20-25°C.</i>
Humidity range <i>Phạm vi độ ẩm</i>	Prolong storage humidity shall not exceed 50%. <i>Độ ẩm kho được kỳ vọng không vượt quá 50%.</i>
Sunlight effect <i>Ảnh hưởng của ánh nắng Mặt Trời</i>	Keep away from sunlight. Sunlight could cause discoloration and long exposure could severely reduce physical properties of gloves. <i>Giữ tránh ánh nắng mặt trời. Ánh nắng có thể là nguyên nhân làm mất màu sắc và phơi bày trong thời gian dài có thể làm giảm mạnh tính chất vật lý của găng tay.</i>

BUREAU VERITAS
Certification



NAM VIET GLOVE JOINT STOCK COMPANY

HEAD OFFICE: 23A NGUYEN HUU CANH STREET, WARD 22,
BINH THANH DISTRICT, HO CHI MINH CITY, VIETNAM

This is a multi-site certificate, additional site(s) are listed on the next page(s)

Bureau Veritas Certification Holding SAS – UK Branch certifies that the
Management System of the above organization has been audited and found
to be in accordance with the requirements of the Management System standards
detailed below.

Standard

ISO 9001:2015

Scope of certification

MANUFACTURE OF NON STERILE POWDER FREE LATEX
AND NITRILE EXAMINATION GLOVES FOR USE IN THE MEDICAL,
FOOD AND INDUSTRIAL SECTORS

Original cycle start date: 01 March 2016
Expiry date of previous cycle: 28 February 2019
Recertification Audit date: 23 August 2018
Recertification cycle start date: 14 September 2018
Subject to the continued satisfactory operation of the organization's Management
System, this certificate expires on: 28 February 2022

Certificate no.: VN.4277793/Q Version: 1 Revision date: 14 September 2018

NGUYEN TU HAI



0008

Certification body address: 5th Floor, 66 Prescott Street, London, E1 8HQ, United Kingdom
Local Office: Unit 4-4A, 4th Floor, E-Town 1 Building, 354 Cong Hoa Street,
Tan Binh District, Ho Chi Minh City, Vietnam

Further clarifications regarding the scope of this certificate and the applicability of the
Management System requirements may be obtained by consulting the organization.
To check this certificate validity, please call +84-20-3912 2246.

Page 1 of 2

BUREAU VERITAS
Certification



NAM VIET GLOVE JOINT STOCK COMPANY

Standard

ISO 9001:2015

Scope of certification

HEAD OFFICE: MANUFACTURING MANAGEMENT OF NON STERILE
POWDER FREE LATEX AND NITRILE EXAMINATION GLOVES
FOR USE IN THE MEDICAL, FOOD AND INDUSTRIAL SECTORS

Site Name/ Location	Site addition Date	Site Address	Site Scope
FACTORY	01 March 2016	HAMLET 7, PHUOC BINH COMMUNE, LONG THANH DISTRICT, DONG NAI PROVINCE, VIETNAM	MANUFACTURE OF NON STERILE POWDER FREE LATEX AND NITRILE EXAMINATION GLOVES FOR USE IN THE MEDICAL, FOOD AND INDUSTRIAL SECTORS

Certificate no.: VN.4277793/Q Version: 1 Revision date: 14 September 2018

NGUYEN TU HAI



0008

Certification body address: 5th Floor, 66 Prescott Street, London, E1 8HQ, United Kingdom
Local Office: Unit 4-4A, 4th Floor, E-Town 1 Building, 354 Cong Hoa Street,
Tan Binh District, Ho Chi Minh City, Vietnam

Further clarifications regarding the scope of this certificate and the applicability of the
Management System requirements may be obtained by consulting the organization.
To check this certificate validity, please call +84-20-3912 2246.

Page 2 of 2

BUREAU VERITAS
Certification



NAM VIET GLOVE JOINT STOCK COMPANY

Head Office and Operative Site:

23A Nguyen Huu Canh, Binh Thanh District, Ho Chi Minh City, Vietnam

Operative Unit:

Hamlet 7, Phuoc Binh Commune, Long Thanh District, Dong Nai Province, Vietnam

Bureau Veritas Italia S.p.A. certify that the Management System of the above organisation has been audited and found to be in accordance with the requirements of the management system standards detailed below

Standard

EN ISO 13485:2016

Scope of certification

Manufacture of non sterile powder free latex and nitrile examination gloves.

Certification awarded in conformity with the requirements of ACCREDIA DT 02-DC REV.00

Certification cycle start date: **31 January 2019**

Subject to the continued satisfactory operation of the organization's Management System, this certificate expires on: **01 February 2022**

Original certification date: **02 February 2016**

Certificate No. **IT287017**

Version **N.1** Revision date: **31 January 2019**


ANDREA FILIPPI - Local Technical Manager

Certification body address:
Bureau Veritas Italia spa, Viale Monza, 347, 20126 Milano, Italia

Further clarifications regarding the scope of this certificate and the applicability of the management system requirements may be obtained by consulting the organisation.
To check this certificate validity please refer to the website
<http://www.bureauveritas.it/certificate>



SGS	N° 009A	SGE	N° 009M
SGA	N° 009D	EGAS	N° 004P
SGD	N° 009B	GHS	N° 008D
SGE	N° 009F	SGP	N° 006E
PSHS	N° 0031	SSI	N° 013G
PSB	N° 076C		

Member degli Accordi di Mutuo Riconoscimento ISO e IAF
Signatory of EA and IAF Mutual Recognition Agreements



Notified Body 0321

Issued to: **Nam Viet Glove Joint Stock Company**
23A Nguyen Huu Canh
Ward 22, Binh Thanh District
Ho Chi Minh City
Vietnam

SATRA Client : P1727

EC Type-Examination Certificate

Number 8915 Issue 1

Date first issued: 03/08/2017

This is to certify that the product group reference "NPF-FT30-24, NPF-FT35-24, NAG-FT37-24, NPF-FT40-24, NPF-FT55-24" comprising the following products:

Product Reference	Description		
NPF-FT30-24	Five Finger Powder Free Nitrile Gloves		
NPF-FT35-24			
NAG-FT37-24			
NPF-FT40-24			
NPF-FT55-24BK			
Classification			
Sizes: S, M, L XL	EN ISO 374-1:2016 (Type C)	Level	EN 374-4:2013
	Sodium hydroxide 40%	6	11.8%
	Sulphuric acid 50%	6	4.4%
	Formaldehyde 37%	2	43.0%
	EN 374-5:2016		
	Protection against bacteria and fungi	Pass	
	Protection against viruses	N/A	

Technical reports:

SATRA: SPC0257773/1721, CHM0255399/1711/SMcD, CHM0257912/1721/SMcD/B, CHM0257912/1721/SMcD/A

has been subject to an EC Type-examination in accordance with Article 10 of the PPE Directive (89/686/EEC) and has been shown to satisfy the relevant provisions of this Directive for the complex category through:

- Testing to the following standard: **EN ISO 374-1:2016; EN420:2003+A1:2009**
- Examination of the relevant technical documentation.

You are therefore licensed to mark the product(s) listed above in accordance with Article 13 of Directive (89/686/EEC) and any relevant amending Directives once you have drawn up an EC declaration of product conformity. Please note that:

- Full details of the certification and product are contained in the manufacturer's technical file
- This certificate is issued subject to the conditions on the reverse side of this certificate
- CE Marking of production is also reliant on current compliance with Directive 89/686/EEC Article 11
- Where a translation of this certificate exists, the English language version shall be considered as the authoritative text

Signed:

(Tara Holly)

Date 03/08/2017

Signed:

On behalf of SATRA

CONDITIONS AND NOTES

1. CARE AND PROTECTION OF CERTIFICATES

This certificate shall be kept together with the relevant technical files in a safe place by the client named on this certificate. Production of this certificate and other documentation may be required by a representative of the EC member state government (in the UK, this will be Trading Standards Department of Local Authorities) and these must be produced within the specified time.

2. EXPIRY OF CERTIFICATES

This certificate relates only to the condition of the testable items at the time of the certification procedure. This certificate will cover the product until such a time that any of these conditions are changed. The certificate therefore does not have an expiry date.

3. WITHDRAWAL OF CERTIFICATES

SATRA Technology Centre reserves the right to withdraw this certificate if it is found that a condition of manufacture, design, materials or packaging have been changed and therefore no longer comply with the requirements of the PPE directive.

4. LOSS OF CERTIFICATES

If you lose this certificate you can get a duplicate from SATRA Technology Centre for a minimal charge by quoting details such as the approximate date of issue and the product reference.

PPE Doc 13/14 Issue 1

SATRA Technology Centre, Wyndham Way, Telford Way,
Kettering, Northamptonshire, NN16 8SD, United Kingdom
Tel: +44 (0)1536 410000 Fax: +44 (0)1536 410626
email: info@satra.co.uk www.satra.co.uk



SATRA Technology Centre Ltd
Wyndham Way, Telford Way, Kettering,
Northamptonshire, NN16 8SD United Kingdom
Tel: +44 (0) 1533 410000
Fax +44 (0) 1533 410626
email: info@satra.com
www.satra.com



Customer details: Nam Viet Glove Joint Stock Company
23a Nguyen Huu Canh St
Ward 22
Binh Thanh District
HCMC
Vietnam

SATRA reference: CHM0255399/1711/
SMcD

Your reference:

Date of report: 19th April 2017

Samples received: 20th March 2017

For the attention of: Ooi Loon Seng

Date(s) work
carried out: 28th March to 18th April
2017

TECHNICAL REPORT

Subject: BS EN 16523-1:2015 resistance to permeation by chemicals on gloves
described as Nitrile Powder Free Gloves, Product code: NPF-FT30-24LB-03
against 40% sodium hydroxide, 50% sulphuric acid and 37% formaldehyde

Conditions of Issue:

This report may be forwarded to other parties provided that it is not changed in any way. It must not be published, for example by including it in advertisements, without the prior, written permission of SATRA.

Results given in this report refer only to the samples submitted for analysis and tested by SATRA. Comments are for guidance only.

Tests marked \neq fall outside the UKAS Accreditation Schedule for SATRA. All interpretations of results of such tests and the comments based upon them are outside the scope of UKAS accreditation and are based on current SATRA knowledge.

A satisfactory test report in no way implies that the product tested is approved by SATRA and no warranty is given as to the performance of the product tested. SATRA shall not be liable for any subsequent loss or damage incurred by the client as a result of information supplied in the report.

The uncertainty of the results (UoM) in this report is based on a standard uncertainty multiplied by a coverage factor $k=2$, which provides for a confidence level of approximately 95%.

Report signed by: S McDonald
Position: Chemical Technologist
Department: Chemical & Analytical Technology

(Page 1 of 7)

WORK REQUESTED:

Samples of gloves described as Nitrile Powder Free Gloves, Product code: NPF-FT30-24LB-03 were received on the 20th March 2017 for testing in accordance with BS EN 16523-1:2015 and assessment in accordance with the requirements of BS EN ISO 374-1: 2016.

CONCLUSION:

When assessed in accordance with the requirements of BS EN ISO 374-1:2016 the samples of gloves described as Nitrile Powder Free Gloves, Product code: NPF-FT30-24LB-03 achieved the following performance levels:

Chemical	Performance level
40% sodium hydroxide (CAS: 1310-73-2)	6
50% sulphuric acid (CAS: 7664-93-9)	6
37% formaldehyde (CAS: 50-00-0)	2

Full results are reported in the following tables.

TESTING REQUIRED:

- BS EN 16523-1:2015 - Determination of material resistance to permeation by chemicals. Part 1: Permeation by liquid chemical under conditions of continuous contact


RESULTS AND REQUIREMENTS:

BS EN ISO 374-1:2016 - Protective gloves against dangerous chemicals and micro-organisms. Part 1: Terminology and performance requirements for chemical risks. Table 1: Permeation performance levels.


Permeation performance level	Measured breakthrough time (minutes)
1	>10
2	>30
3	>60
4	>120
5	>240
6	>480

Performance levels are based on the lowest individual result achieved per chemical.


Test/Property	Sample reference:	Nitrile Powder Free Gloves, Product code: NPF-FT30-24LB-03		Performance
BS EN 16523-1:2015 in accordance with SATRA SOP CAT-009 Using PTFE permeation cells with standardised dimensions	Test information:	Chemical: 40% sodium hydroxide		Level 6
		Normalised permeation rate (NPR): 1 µg/cm²/min		
		Detection technique: Conductimetry (continuous measurement)		
		Collection medium: Deionised water (closed loop)		
		Collection medium stirring rate: 45 – 65 ml/min (each cell constant to within ± 10%)		
		Test temperature: (23 ± 1) °C		
	Specimen	Thickness (mm)Δ	Breakthrough time (mins)	
	1	0.07	>480	
	2	0.06	>480	
	3	0.06	>480	
	Test result:	>480		
	UoM:	± <1		
Visual appearance of specimens after testing:		Discoloured		

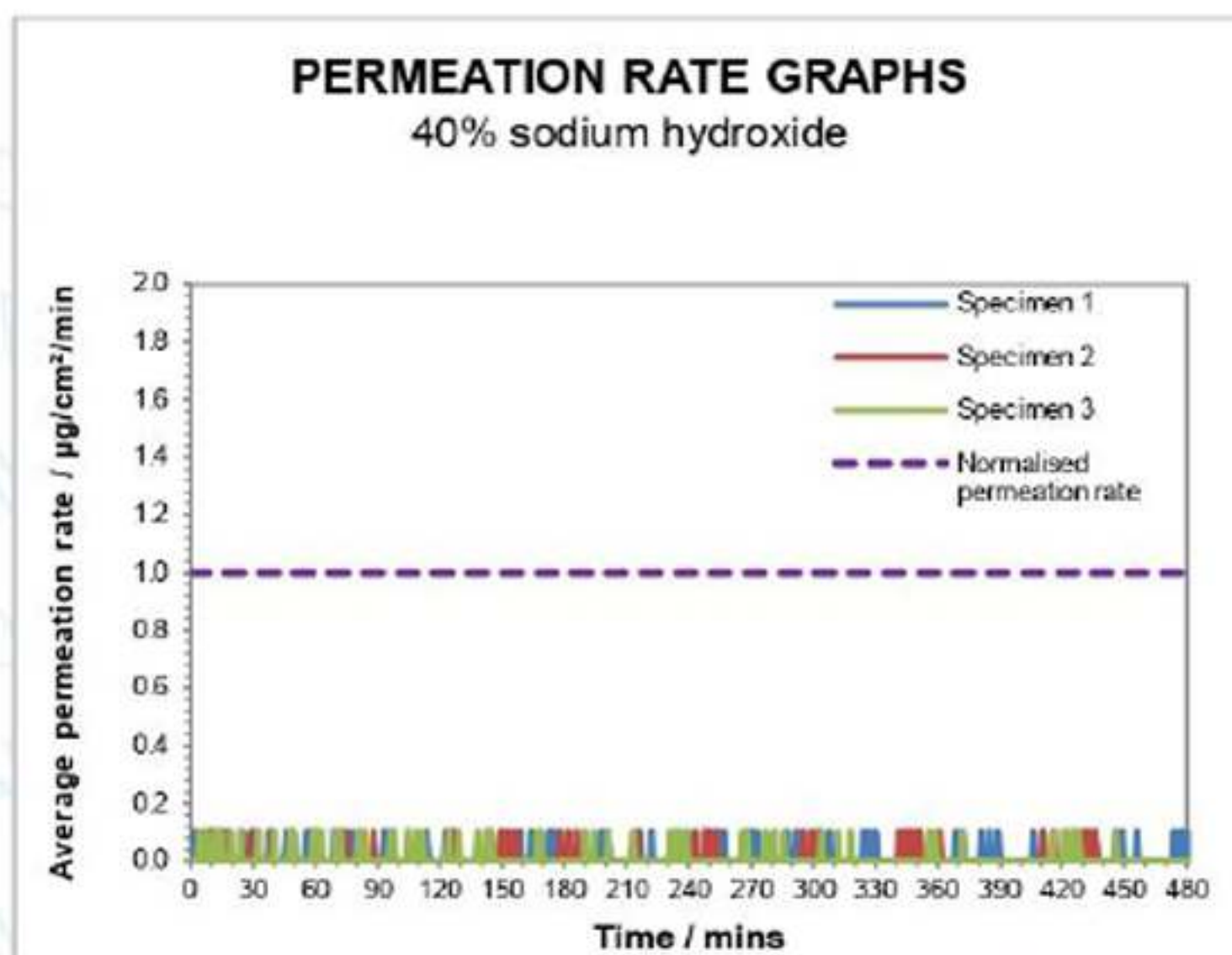

Test/Property	Sample reference:	Nitrile Powder Free Gloves, Product code: NPF-FT30-24LB-03		Performance
BS EN 16523-1:2015 in accordance with SATRA SOP CAT-009	Test information:	Chemical: 50% sulphuric acid		Level 6
		Normalised permeation rate (NPR): 1 µg/cm²/min		
		Detection technique: Conductimetry (continuous measurement)		
		Collection medium: Deionised water (closed loop)		
		Collection medium stirring rate: 45 – 65 ml/min (each cell constant to within ± 10%)		
		Test temperature: (23 ± 1) °C		
	Using PTFE permeation cells with standardised dimensions	Specimen	Thickness (mm)△	
1		0.06	>480	
2		0.06	>480	
3		0.06	>480	
		Test result:	>480	
		UoM:	± <1	
Visual appearance of specimens after testing:		Discoloured		

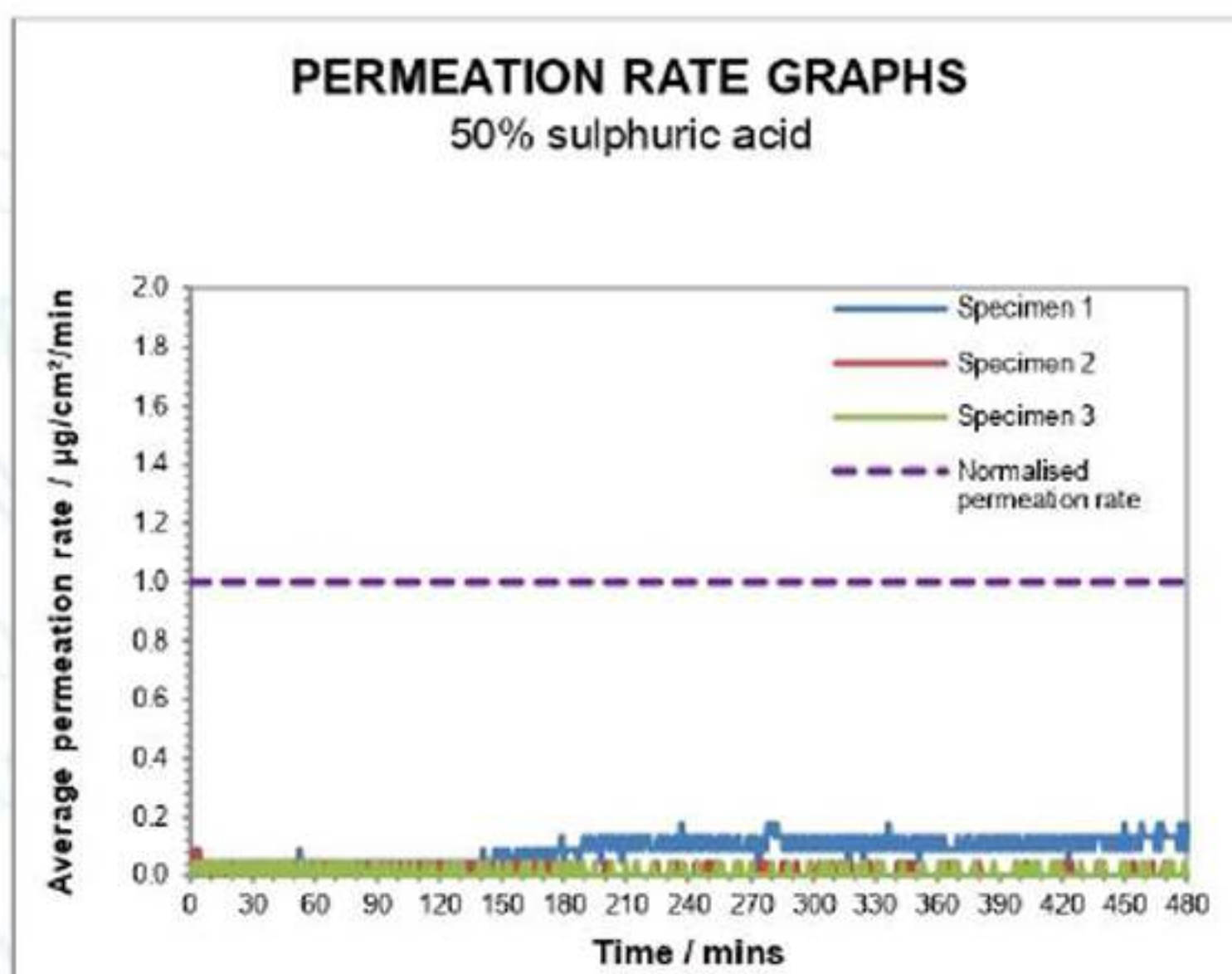

Test/Property	Sample reference:	Nitrile Powder Free Gloves, Product code: NPF-FT30-24LB-03		Performance
BS EN 16523-1:2015 in accordance with SATRA SOP CAT-025	Test information:	Chemical: 37% formaldehyde		Level 2
		Normalised permeation rate (NPR): 1 µg/cm²/min		
		Detection technique: Spectrophotometry (periodic measurement)		
		Collection medium: Deionised water (closed loop)		
		Collection medium stirring rate: 45 – 65 ml/min (each cell constant to within ± 10%)		
		Test temperature: (23 ± 1) °C		
	Using PTFE permeation cells with standardised dimensions	Specimen	Thickness (mm)△	
1		0.07	Between 91 and 120	
2		0.07	Between 91 and 120	
3		0.06	Between 46 and 60	
		Test result:	Between 46 and 60	
	UoM:	See note below		
Visual appearance of specimens after testing:		No change		

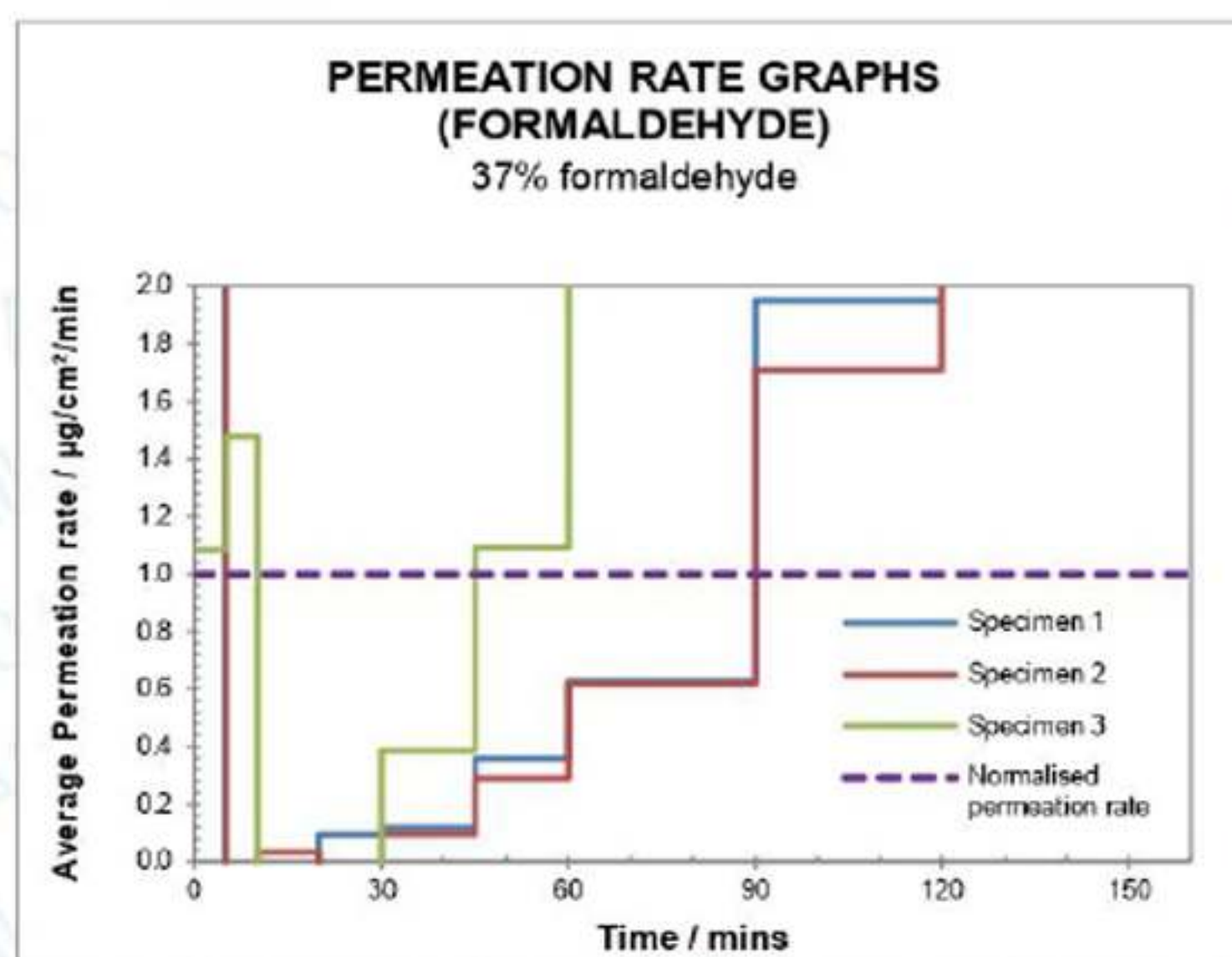

The high readings at 0-10 minutes were deemed to be outliers and do not represent chemical breakthrough.

Formaldehyde is determined by discrete sampling; therefore the permeation rate graph is not a smooth curve.

For SOP CAT-025, where both the P_1 and P_u are observed in the same sampling range, uncertainty is expressed as the time difference between the mid-point of the range and the previous sampling time. This uncertainty is included in the reported result.

△ BS EN 16523-1:2015 does not require the test specimen thicknesses to be reported, this information is indicative only.

▼ Breakthrough expressed as a range between discrete sampling points where the average permeation rate exceeds the NPR. Due to the complexity of the detection technique, the minimum sampling frequency as specified in table 1 of BS EN 16523-1:2015 is not possible.

APPENDICES:



Gloves described as Nitrile Powder Free Gloves,
Product code: NPF-FT30-24LB-03

TERMS AND CONDITIONS OF BUSINESS

1. **GENERAL**
Work done or services undertaken are subject to the terms and conditions detailed below and all other conditions, warranties and representations, expressed or implied are hereby excluded.
2. **PRICES**
Prices are based on current material and production costs, exchange rates, duty and freight and are subject to change without notice.
3. **DELIVERY ESTIMATES**
Delivery estimates are made in good faith and date from receipt of a written order and full information to enable us to proceed. While SATRA or its subsidiaries (hereafter referred to as "SATRA") make every effort to fulfil them, such estimates are subject to unforeseen events and if not maintained, cannot give rise to any claim. Offers "ex stock" are subject to prior sale.
4. **CANCELLATION AND RETURNS**
Cancellation of orders for goods, services, training or consultancy is only acceptable by prior agreement of SATRA and a charge will normally be made.
5. **CLAIMS**
Claims for errors, shortages etc should be notified within 10 days of date of receipt. In the event of goods damaged in transit, packing materials should be retained for examination; otherwise no liability can be accepted.
6. **PAYMENT TERMS**
Payment terms are net 21 days from date of invoice. Failure to comply with the terms of payment may result in delayed delivery of goods and services and a review of the Customer's credit account. Should the customer become subject to an administration order, or becomes bankrupt or goes into liquidation, SATRA has a right to cancel any contract and discontinue any work. SATRA reserves the right to adjust US Dollar and Euro sales price where customer exceeds credit terms and where the exchange rate has moved more than 10% since invoicing.
7. **RETENTION OF TITLE**
All goods remain the property of SATRA until paid in full. Under no circumstances will a customer's purchase order override SATRA's Retention of Title clause. In the case of software, the ownership of the software remains with SATRA. Payment of invoices in full will entitle the customer to use the software under licence until (a) they cease to be a member of SATRA or (b) they cease trading. In both instances, the licence shall then revert to SATRA.
8. **GUARANTEE**
All goods manufactured by SATRA are guaranteed both as regards material and workmanship. Any part returned carriage paid, within twelve months from date of supply and found defective, will be repaired or replaced at SATRA's option free of charge. SATRA admits no liability for loss, damage or delay consequent on any defect in any goods supplied by SATRA.
9. **TEST REPORTS**
Results given in test reports refer only to samples submitted for analysis and tested by SATRA. A satisfactory test report in no way implies that the product tested is approved by SATRA and no warranty is given as to the performance of the product tested. SATRA shall not be liable for any subsequent loss or damage incurred by the Customer as a result of information supplied in a test report.
10. **TEST SAMPLES**
Unless otherwise agreed in advance, test samples will be disposed of 6 weeks after the date of the final report. If required, samples can be returned at the Customer's expense.
11. **RESPONSIBILITY**
Every effort is made to ensure accuracy in description, drawings and other information in correspondence, catalogues, etc but no warranty is given in this respect and SATRA shall not be liable for any error therein. SATRA carries out all tests and/or advises only on the basis that the same are carried out, made or given without any responsibility whether for negligence or otherwise. SATRA and its servants or agents will not be liable for any damage or loss direct or indirect of whatsoever kind, whether or not the same results directly or indirectly from negligence on the part of SATRA or its servants or agents.
12. **CONFIDENTIALITY**
Unless specifically excluded in the terms of an individual contract between SATRA and its Customer, the following shall apply to all reports, advice, drawings, photographs, specifications or data:
 - i. The above shall not be disclosed to third parties or used in litigation without the consent of SATRA.
 - ii. Where SATRA has given consent to disclosure, the Customer shall draw the attention of the third party to these terms of business and the basis on which SATRA undertakes test, reporting and advising. The Customer shall indemnify SATRA for any failure to do so.
 - iii. The above items are submitted to the Customer as confidential documents. Confidentiality shall continue to apply after completion of the business, but shall cease to apply to information or knowledge which may come into the public domain.
13. **CONSTRUCTION AND ARBITRATION**
The laws of England shall govern all contracts and the parties submit to exclusive jurisdiction of the courts of England, unless otherwise agreed.

Issue Date: 1st October 2009

Nam Viet Glove Joint Stock Company
SATRA Reference: CHM0255399/1711/SMcD
Date: 19th April 2017

Signed:

A handwritten signature in black ink that reads "S. McDonald".



SATRA Technology Centre Ltd
Wyndham Way, Telford Way, Kettering,
Northamptonshire, NN16 8SD United Kingdom
Tel: +44 (0) 1536 410000
Fax +44 (0) 1536 410626
email: info@satra.com
www.satra.com



Customer details: Nam Viet Glove Joint Stock Company
23a Nguyen Huu Canh St
Ward 22
Binh Thanh District
HCMC
Vietnam

SATRA reference: CHM0257912/1721/
SMcD/A

Your reference:

Date of report: 9th June 2017

Samples received: 16th May 2017

For the attention of: Ooi Loon Seng

Date(s) work
carried out: 2nd to 8th June 2017

TECHNICAL REPORT

Subject: Chemical innocuousness testing in accordance with BS EN 420: 2003 + A1:
2009 on gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue

Conditions of Issue:

This report may be forwarded to other parties provided that it is not changed in any way. It must not be published, for example by including it in advertisements, without the prior, written permission of SATRA.

Results given in this report refer only to the samples submitted for analysis and tested by SATRA. Comments are for guidance only.

Tests marked \neq fall outside the UKAS Accreditation Schedule for SATRA. All interpretations of results of such tests and the comments based upon them are outside the scope of UKAS accreditation and are based on current SATRA knowledge.

A satisfactory test report in no way implies that the product tested is approved by SATRA and no warranty is given as to the performance of the product tested. SATRA shall not be liable for any subsequent loss or damage incurred by the client as a result of information supplied in the report.

The uncertainty of the results (UoM) in this report is based on a standard uncertainty multiplied by a coverage factor $k=2$, which provides for a confidence level of approximately 95%.

Report signed by: S McDonald
Position: Chemical Technologist
Department: Chemical & Analytical Technology

(Page 1 of 4)

WORK REQUESTED:

Samples of gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue were received on the 16th May 2017 for testing in accordance with the innocuousness requirements of BS EN 420:2003 + A1:2009 Protective Gloves - General Requirements and Test Methods.

MATERIALS TESTED:

- NPF-FT30-24LB Powder free nitrile gloves-blue

CONCLUSIONS:

The samples of gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue were assessed in accordance with the innocuousness requirements of BS EN 420:2003 + A1:2009 and were found to meet with the requirement for pH value.

As required under clause 4.3.1 General innocuousness of protective gloves, the samples of gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue were found to meet the REACH annex XVII requirement for PAHs.

Full results are reported in the following tables.

TESTS REQUIRED:

- BS EN 420:2003 Clause 4.3.2 pH Value (ISO 3071 for Textiles & other materials & ISO 4045 for leathers)
- #SATRA SOP CAT-018 – Determination of PAHs (based on ZEK 01.4-08)

RESULTS AND REQUIREMENTS:

BS EN 420:2003 + A1: 2009 Clause 4.3.2 pH value

Date of determination: 2nd June 2017

Sample	Method	pH Value	UoM	Pass/Fail
NPF-FT30-24LB Powder free nitrile gloves-blue	ISO 3071:2005 (water extraction)	7.6	± 0.1	Pass
Requirement	pH value greater than 3.5 and less than 9.5			

The extraction solution temperature was 25°C and at pH 7.1

#SATRA SOP CAT-018 – Determination of PAHs

Analysed by Gas Chromatography with Mass Spectrometry (GC-MS)

Sample	PAHs detected (mg/kg)	Pass/Fail
NPF-FT30-24LB Powder free nitrile gloves-blue	<0.2 (of each PAH listed in the appendices)	Pass
Requirements: REACH 1907/2006 annex XVII entry number 50	<1mg/kg of each PAH listed in the appendices	-

APPENDICES:

REACH Regulation (EC) No 1907/2006 Annex XVII entry number 50 as amended by regulation 1272/2013

PAH	CAS Number	Requirements
Benzo[a]anthracene	56-55-3	Articles shall not be placed on the market for supply to the general public, if any of their rubber or plastic components that come into direct as well as prolonged or short-term repetitive contact with the human skin or the oral cavity, under normal or reasonably foreseeable conditions of use, contain more than 1 mg/kg of any of the listed PAHs.
Chrysene	218-01-9	
Benzo[b]fluoranthene	205-99-2	
Benzo[k]fluoranthene	207-08-9	
Benzo[j]fluoranthene	205-82-3	
Benzo[e]pyrene	192-97-2	
Benzo[a]pyrene	50-32-8	
Dibenzo[a,h]anthracene	53-70-3	



Gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue


TERMS AND CONDITIONS OF BUSINESS

1. **GENERAL**
Work done or services undertaken are subject to the terms and conditions detailed below and all other conditions, warranties and representations, expressed or implied are hereby excluded.
2. **PRICES**
Prices are based on current material and production costs, exchange rates, duty and freight and are subject to change without notice.
3. **DELIVERY ESTIMATES**
Delivery estimates are made in good faith and date from receipt of a written order and full information to enable us to proceed. While SATRA or its subsidiaries (hereafter referred to as "SATRA") make every effort to fulfil them, such estimates are subject to unforeseen events and if not maintained, cannot give rise to any claim. Offers "ex stock" are subject to prior sale.
4. **CANCELLATION AND RETURNS**
Cancellation of orders for goods, services, training or consultancy is only acceptable by prior agreement of SATRA and a charge will normally be made.
5. **CLAIMS**
Claims for errors, shortages etc should be notified within 10 days of date of receipt. In the event of goods damaged in transit, packing materials should be retained for examination; otherwise no liability can be accepted.
6. **PAYMENT TERMS**
Payment terms are net 21 days from date of invoice. Failure to comply with the terms of payment may result in delayed delivery of goods and services and a review of the Customer's credit account. Should the customer become subject to an administration order, or becomes bankrupt or goes into liquidation, SATRA has a right to cancel any contract and discontinue any work. SATRA reserves the right to adjust US Dollar and Euro sales price where customer exceeds credit terms and where the exchange rate has moved more than 10% since invoicing.
7. **RETENTION OF TITLE**
All goods remain the property of SATRA until paid in full. Under no circumstances will a customer's purchase order override SATRA's Retention of Title clause. In the case of software, the ownership of the software remains with SATRA. Payment of invoices in full will entitle the customer to use the software under licence until (a) they cease to be a member of SATRA or (b) they cease trading. In both instances, the licence shall then revert to SATRA.
8. **GUARANTEE**
All goods manufactured by SATRA are guaranteed both as regards material and workmanship. Any part returned carriage paid, within twelve months from date of supply and found defective, will be repaired or replaced at SATRA's option free of charge. SATRA admits no liability for loss, damage or delay consequent on any defect in any goods supplied by SATRA.
9. **TEST REPORTS**
Results given in test reports refer only to samples submitted for analysis and tested by SATRA. A satisfactory test report in no way implies that the product tested is approved by SATRA and no warranty is given as to the performance of the product tested. SATRA shall not be liable for any subsequent loss or damage incurred by the Customer as a result of information supplied in a test report.
10. **TEST SAMPLES**
Unless otherwise agreed in advance, test samples will be disposed of 6 weeks after the date of the final report. If required, samples can be returned at the Customer's expense.
11. **RESPONSIBILITY**
Every effort is made to ensure accuracy in description, drawings and other information in correspondence, catalogues, etc but no warranty is given in this respect and SATRA shall not be liable for any error therein. SATRA carries out all tests and/or advises only on the basis that the same are carried out, made or given without any responsibility whether for negligence or otherwise. SATRA and its servants or agents will not be liable for any damage or loss direct or indirect of whatsoever kind, whether or not the same results directly or indirectly from negligence on the part of SATRA or its servants or agents.
12. **CONFIDENTIALITY**
Unless specifically excluded in the terms of an individual contract between SATRA and its Customer, the following shall apply to all reports, advice, drawings, photographs, specifications or data:
 - i. The above shall not be disclosed to third parties or used in litigation without the consent of SATRA.
 - ii. Where SATRA has given consent to disclosure, the Customer shall draw the attention of the third party to these terms of business and the basis on which SATRA undertakes test, reporting and advising. The Customer shall indemnify SATRA for any failure to do so.
 - iii. The above items are submitted to the Customer as confidential documents. Confidentiality shall continue to apply after completion of the business, but shall cease to apply to information or knowledge which may come into the public domain.
13. **CONSTRUCTION AND ARBITRATION**
The laws of England shall govern all contracts and the parties submit to exclusive jurisdiction of the courts of England, unless otherwise agreed.

Issue Date: 1st October 2009

Nam Viet Glove Joint Stock Company
SATRA Reference: CHM0257912/1721/SMcD/A
Date: 9th June 2017

Signed:

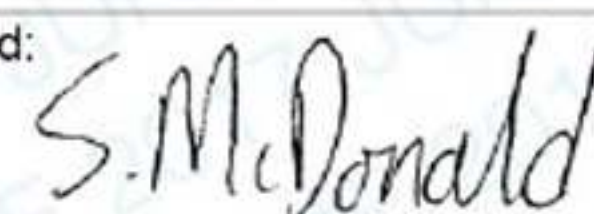

(Page 4 of 4)



SATRA Technology Centre Ltd
Wyndham Way, Telford Way, Kettering,
Northamptonshire, NN16 8SD United Kingdom
Tel: +44 (0) 1536 410000
Fax +44 (0) 1536 410626
email: info@satra.com
www.satra.com



Customer details: Nam Viet Glove Joint Stock Company
23a Nguyen Huu Canh St
Ward 22
Binh Thanh District
HCMC
Vietnam

SATRA reference: CHM0257912/1721/
SMcD/B

Your reference:

Date of report: 9th June 2017

Samples received: 16th May 2017

For the attention of: Ooi Loon Seng

Date(s) work
carried out: 6th June 2017

TECHNICAL REPORT

Subject: BS EN 374-4: 2013 determination of resistance to degradation by chemicals on
gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue

Conditions of Issue:

This report may be forwarded to other parties provided that it is not changed in any way. It must not be published, for example by including it in advertisements, without the prior, written permission of SATRA.

Results given in this report refer only to the samples submitted for analysis and tested by SATRA. Comments are for guidance only.

Tests marked \neq fall outside the UKAS Accreditation Schedule for SATRA. All interpretations of results of such tests and the comments based upon them are outside the scope of UKAS accreditation and are based on current SATRA knowledge.

A satisfactory test report in no way implies that the product tested is approved by SATRA and no warranty is given as to the performance of the product tested. SATRA shall not be liable for any subsequent loss or damage incurred by the client as a result of information supplied in the report.

The uncertainty of the results (UoM) in this report is based on a standard uncertainty multiplied by a coverage factor $k=2$, which provides for a confidence level of approximately 95%.

Report signed by: S McDonald
Position: Chemical Technologist
Department: Chemical & Analytical Technology

(Page 1 of 5)

WORK REQUESTED:

Samples of gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue were received on the 16th May 2017 for testing in accordance with BS EN 374-4:2013. Full results are given below.

CONCLUSION:

When assessed in accordance with BS EN 374-4:2013 the samples of gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue achieved the following degradation results:

Chemical	Mean degradation / %
Formaldehyde (CAS: 50-00-0)	43.0
40% Sodium hydroxide (CAS: 1310-73-2)	11.8
96% Sulphuric acid (CAS: 7664-93-9)	100.0

NOTE: The quoted mean degradation values are subject to often significant measurement uncertainties. Please see results tables below for more information

TESTING REQUIRED:

- BS EN 374-4:2013. Protective gloves against chemicals and micro-organisms. Part 4: Determination of resistance to degradation by chemicals.

RESULTS:

Sample description:	NPF-FT30-24LB Powder free nitrile gloves-blue		
Challenge chemical:	Formaldehyde (CAS: 50-00-0)		
Test temperature / °C:	(23 ± 1)		
Degradation / %:	Glove 1	Glove 2	Glove 3
	41.7	56.1	31.1
Mean degradation (DR) / %:	43.0		
Standard deviation (σ_{DR}) / %:	12.5		
UoM \pm / %:	21.5		
Appearance of samples after testing:	No change		

Sample description:	NPF-FT30-24LB Powder free nitrile gloves-blue		
Challenge chemical:	40% Sodium hydroxide (CAS: 1310-73-2)		
Test temperature / °C:	(23 ± 1)		
Degradation / %:	Glove 1	Glove 2	Glove 3
	-14.2	34.9	14.7
Mean degradation (DR) / %:	11.8		
Standard deviation (σ_{DR}) / %:	24.7		
UoM \pm / %:	27.1		
Appearance of samples after testing:	No change		

Sample description:	NPF-FT30-24LB Powder free nitrile gloves-blue		
Challenge chemical:	96% Sulphuric acid (CAS: 7664-93-9)		
Test temperature / °C:	(23 ± 1)		
Degradation / %:	Glove 1	Glove 2	Glove 3
	100.0	100.0	100.0
Mean degradation (DR) / %:	100.0		
Standard deviation (σ_{DR}) / %:	0.0		
UoM \pm / %:	16.7		
Appearance of samples after testing:	Discoloured and degraded		

♦ Absolute measurement uncertainty of the mean degradation value; it is therefore inferred that the true degradation value, with 95% confidence, lies within the range (DR ± UoM) %.

NOTE: Where the test specimens gave an increased puncture force after chemical exposure, the result is reported as a negative degradation.

APPENDICES:



Gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue

TERMS AND CONDITIONS OF BUSINESS

1. **GENERAL**
Work done or services undertaken are subject to the terms and conditions detailed below and all other conditions, warranties and representations, expressed or implied are hereby excluded.
2. **PRICES**
Prices are based on current material and production costs, exchange rates, duty and freight and are subject to change without notice.
3. **DELIVERY ESTIMATES**
Delivery estimates are made in good faith and date from receipt of a written order and full information to enable us to proceed. While SATRA or its subsidiaries (hereafter referred to as "SATRA") make every effort to fulfil them, such estimates are subject to unforeseen events and if not maintained, cannot give rise to any claim. Offers "ex stock" are subject to prior sale.
4. **CANCELLATION AND RETURNS**
Cancellation of orders for goods, services, training or consultancy is only acceptable by prior agreement of SATRA and a charge will normally be made.
5. **CLAIMS**
Claims for errors, shortages etc should be notified within 10 days of date of receipt. In the event of goods damaged in transit, packing materials should be retained for examination; otherwise no liability can be accepted.
6. **PAYMENT TERMS**
Payment terms are net 21 days from date of invoice. Failure to comply with the terms of payment may result in delayed delivery of goods and services and a review of the Customer's credit account. Should the customer become subject to an administration order, or becomes bankrupt or goes into liquidation, SATRA has a right to cancel any contract and discontinue any work. SATRA reserves the right to adjust US Dollar and Euro sales price where customer exceeds credit terms and where the exchange rate has moved more than 10% since invoicing.
7. **RETENTION OF TITLE**
All goods remain the property of SATRA until paid in full. Under no circumstances will a customer's purchase order override SATRA's Retention of Title clause. In the case of software, the ownership of the software remains with SATRA. Payment of invoices in full will entitle the customer to use the software under licence until (a) they cease to be a member of SATRA or (b) they cease trading. In both instances, the licence shall then revert to SATRA.
8. **GUARANTEE**
All goods manufactured by SATRA are guaranteed both as regards material and workmanship. Any part returned carriage paid, within twelve months from date of supply and found defective, will be repaired or replaced at SATRA's option free of charge. SATRA admits no liability for loss, damage or delay consequent on any defect in any goods supplied by SATRA.
9. **TEST REPORTS**
Results given in test reports refer only to samples submitted for analysis and tested by SATRA. A satisfactory test report in no way implies that the product tested is approved by SATRA and no warranty is given as to the performance of the product tested. SATRA shall not be liable for any subsequent loss or damage incurred by the Customer as a result of information supplied in a test report.
10. **TEST SAMPLES**
Unless otherwise agreed in advance, test samples will be disposed of 6 weeks after the date of the final report. If required, samples can be returned at the Customer's expense.
11. **RESPONSIBILITY**
Every effort is made to ensure accuracy in description, drawings and other information in correspondence, catalogues, etc but no warranty is given in this respect and SATRA shall not be liable for any error therein. SATRA carries out all tests and/or advises only on the basis that the same are carried out, made or given without any responsibility whether for negligence or otherwise. SATRA and its servants or agents will not be liable for any damage or loss direct or indirect of whatsoever kind, whether or not the same results directly or indirectly from negligence on the part of SATRA or its servants or agents.
12. **CONFIDENTIALITY**
Unless specifically excluded in the terms of an individual contract between SATRA and its Customer, the following shall apply to all reports, advice, drawings, photographs, specifications or data:
 - i. The above shall not be disclosed to third parties or used in litigation without the consent of SATRA.
 - ii. Where SATRA has given consent to disclosure, the Customer shall draw the attention of the third party to these terms of business and the basis on which SATRA undertakes test, reporting and advising. The Customer shall indemnify SATRA for any failure to do so.
 - iii. The above items are submitted to the Customer as confidential documents. Confidentiality shall continue to apply after completion of the business, but shall cease to apply to information or knowledge which may come into the public domain.
13. **CONSTRUCTION AND ARBITRATION**
The laws of England shall govern all contracts and the parties submit to exclusive jurisdiction of the courts of England, unless otherwise agreed.

Issue Date: 1st October 2009

Nam Viet Glove Joint Stock Company
SATRA Reference: CHM0257912/1721/SMcD/B
Date: 9th June 2017

Signed:

S. McDonald



SATRA Technology Centre Ltd
Wyndham Way, Telford Way, Kettering,
Northamptonshire, NN16 8SD United Kingdom
Tel: +44 (0) 1536 410000
Fax +44 (0) 1536 410626
email: info@satra.com
www.satra.com



Customer details: Nam Viet Glove Joint Stock Company
23a Nguyen Huu Canh St
Ward 22
Binh Thanh District
HCMC
Vietnam

SATRA reference: CHM0259430/1727/CL

Your reference:

Date of report: 14th July 2017

Samples received: 16th May 2017

For the attention of: Ooi Loon Seng

Date(s) work carried out: 11th July 2017

TECHNICAL REPORT

Subject: BS EN 374-4:2013 determination of resistance to degradation by chemicals on gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue against 50% Sulphuric acid

Conditions of Issue:

This report may be forwarded to other parties provided that it is not changed in any way. It must not be published, for example by including it in advertisements, without the prior, written permission of SATRA.

Results given in this report refer only to the samples submitted for analysis and tested by SATRA. Comments are for guidance only.

Tests marked \neq fall outside the UKAS Accreditation Schedule for SATRA. All interpretations of results of such tests and the comments based upon them are outside the scope of UKAS accreditation and are based on current SATRA knowledge.

A satisfactory test report in no way implies that the product tested is approved by SATRA and no warranty is given as to the performance of the product tested. SATRA shall not be liable for any subsequent loss or damage incurred by the client as a result of information supplied in the report.

The uncertainty of the results (UoM) in this report is based on a standard uncertainty multiplied by a coverage factor $k=2$, which provides for a confidence level of approximately 95%.

Report signed by: Charlotte Lovell
Position: Chemical Technologist
Department: Chemical & Analytical Technology

(Page 1 of 5)

WORK REQUESTED:

Samples of gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue were received on the 16th May 2017 for testing in accordance with BS EN 374-4:2013. Full results are given below.

CONCLUSION:

When assessed in accordance with BS EN 374-4:2013 the samples of gloves described as NPF-FT30-24LB Powder free nitrile gloves-blue achieved the following degradation results:

Chemical	Mean degradation / %
50% Sulphuric acid (CAS: 7664-93-9)	4.4

NOTE: The quoted mean degradation values are subject to often significant measurement uncertainties. Please see results tables below for more information

TESTING REQUIRED:

- BS EN 374-4:2013. Protective gloves against chemicals and micro-organisms. Part 4: Determination of resistance to degradation by chemicals.

RESULTS:

Sample description:	NPF-FT30-24LB Powder free nitrile gloves-blue		
Challenge chemical:	50% Sulphuric acid		
Test temperature / °C:	(23 ± 1)		
Degradation / %:	Glove 1	Glove 2	Glove 3
	-0.2	14.2	-1.0
Mean degradation (DR) / %:	4.4		
Standard deviation (σ_{DR}) / %:	8.5		
UoM ♦ / ± %:	19.3		
Appearance of samples after testing:	Softened		

♦ Absolute measurement uncertainty of the mean degradation value; it is therefore inferred that the true degradation value, with 95% confidence, lies within the range (DR ± UoM) %.

NOTE: Where the test specimens gave an increased puncture force after chemical exposure, the result is reported as a negative degradation.

APPENDICES:



Sample described as NPF-FT30-24LB Powder free nitrile gloves-blue

TERMS AND CONDITIONS FOR THE SALE OF GOODS AND/OR THE PROVISION OF SERVICES

1. GENERAL

- 1.1 Work done, Services undertaken or the sale of Goods are subject to the terms and conditions detailed below and (subject to clause 5.2) all other conditions, warranties and representations, expressed or implied by statute relating thereto are hereby excluded.
- 1.2 SATRA Technology Centre Limited, its subsidiaries and associated companies (hereinafter referred to as "SATRA") may perform Services for or supply Goods to persons or entities (public, private or governmental) issuing instructions (hereinafter termed the "Client"). Each also known individually as a Party, or jointly as Parties.
- 1.3 These terms and conditions will apply to the Contract between SATRA and the Client to the exclusion of any other terms which the Client may seek to impose or which may be implied by trade, custom, practice or course of dealing.
- 1.4 Unless otherwise agreed in writing no party other than the Client is entitled to provide instructions or information relating to the Goods or Services required or to the delivery of goods, results, reports or certificates.
- 1.5 All references in these terms and conditions to:
 - (a) the "Contract" is the contract between SATRA and the Client for the supply of Goods or Services which is made subject to these terms and conditions; and
 - (b) "Services" are the work or services to be supplied or performed under the Contract (including where relevant the supply of software, components and consumables); and
 - (c) "Goods" are the equipment, consumables or other physical items sold under the Contract (including documents, drawings or other information required in order to operate the equipment).
- 1.6 All drawings, descriptive matter, specifications and advertising material (including brochures and catalogues) are issued or published with the sole purpose of giving an indication of the goods or services being described and shall not form part of the Contract.
- 1.7 Where SATRA and the Client agree that the sale of Goods shall be governed by Incoterms 2010 (or any subsequent revision thereto) then the sale shall be governed by the relevant Incoterms mode of transport which is agreed by SATRA and the Client.

2. FEES AND PAYMENT

- 2.1 Where SATRA has agreed to perform the Services or supply the Goods on the basis of credit then payment terms are net 21 days from date of invoice, unless otherwise specified and may require part payment prior to delivery of the Services or Goods. In the event of the Client failing to make payment as agreed SATRA will be entitled to withhold delivery of the Goods or Services or cancel the Contract. SATRA reserves the right to charge interest on any overdue payments at a rate of 1.5% per month accruing on a daily basis from the date the invoice is due until the date payment is received.
- 2.2 Where the provision of Services or the sale of Goods is subject to a proforma invoice then SATRA shall not be obliged to start working on the provision of the Goods or Services until after payment in full has been made as cleared funds to SATRA.
- 2.3 SATRA reserves the right to charge for any and all expenses incurred as a result of performing the Services required by the Client. Although SATRA will try and provide an estimate of such expenses these may change as a result of circumstances out of SATRA's control.
- 2.4 Unless otherwise agreed in writing, the price for the Goods or Services shall be the price set in the order acknowledgement. SATRA shall not be bound by any price quoted which is not in writing. Prices for the sale of Goods include packing cases and materials but not carriage or installation which will be quoted separately and as agreed with the Client.
- 2.5 Quotations are valid from the date of issue for a period of 90 days unless otherwise specified or agreed in writing.
- 2.6 Should the Client become insolvent, bankrupt, subject to an administration order, enter into liquidation or receivership, or make arrangements with creditors SATRA reserves the right to cancel the Contract and terminate the supply of the Goods or Services. Where the Contract with SATRA is terminated all outstanding monies due from the Client to SATRA shall be immediately payable, and any materials supplied by SATRA to the Client returned. Termination of the Contract shall be without prejudice to any of SATRA's accrued rights.
- 2.7 All invoices issued by SATRA are payable in full. The Client is responsible for payment of withholding and any other taxes and all import duties. Payments made to SATRA shall not be reduced by such amounts.
- 2.8 The Client shall not be entitled to withhold or defer payment due to SATRA as a result of any dispute or counter claim that it may allege against SATRA.
- 2.9 SATRA reserves the right to bring action against the Client in order to collect unpaid fees, including court action. All fees associated with such actions shall be paid for by the Client including legal fees and related costs.
- 2.10 Where unforeseen costs arise as a result of provision of the Goods or carrying out the Services SATRA shall inform the Client immediately but reserves the right to charge additional costs to cover said costs and expenses.

3. INTELLECTUAL PROPERTY RIGHTS

- 3.1 All intellectual property rights belonging to a Party prior to entry into the Contract shall remain with that Party. Nothing in this Contract shall allow transfer of any intellectual property rights from one Party to the other.
- 3.2 In the event of certification services the use of certification marks by the Client may be subject to national and international laws and regulations. The responsibility for the use of these certification marks lies solely with the Client.
- 3.3 All intellectual property rights in reports, drawings, graphs, charts, photographs or any other material (in whatever medium) produced by SATRA pursuant to this Contract shall belong to SATRA. The Client shall have the right to use said material in accordance with the terms of this Contract.
- 3.4 The Client agrees and acknowledges that SATRA retains any and all property rights in concepts, ideas and inventions that may arise during the preparation or provision of any report (including any deliverables provided by SATRA to the Client) and the provision of the Services to the Client.
- 3.5 All intellectual property rights in any software supplied to the Client shall belong to SATRA or SATRA's licensors. With respect to the sale of SATRA Timeline, SATRASUMM and SATRA Visionstitch, provided that the Client is a member of SATRA and has paid its annual Smartcare fee then the Client will be entitled to use the software for its own internal use and will be entitled to receive minor software upgrades and fixes. SATRA may however terminate the supply of software upgrades and fixes for older versions of software which it no longer considers viable to support. The Client's rights to use the software and receive software upgrades and fixes will terminate if the Client has not paid its annual Smartcare fee. Major upgrades are not included within the entitlement to upgrades but may be offered by SATRA from time to time for an additional fee.
- 3.6 SATRA shall observe all statutory provisions with regard to data protection including but not limited to the provisions of the Data Protection Act 1998. To the extent that SATRA processes or gets access to personal data in connection with the Services or otherwise in connection with this Contract, it shall take all reasonable technical and organisational measures to ensure the security of such data (and guard against unauthorised or unlawful processing, accidental loss, destruction or damage to such data).

4. SUSPENSION OR TERMINATION OF SERVICES

- 4.1 Cancellation by the Client of orders for Goods or Services will only be acceptable by prior agreement with SATRA and a charge will usually be made.
- 4.2 SATRA shall not be liable for any delay or failure in providing the Goods or Services due to circumstances beyond its reasonable control (including any failure by the Client to comply with its obligations). If any such circumstances arise which prevent SATRA from delivering the Goods or completing the Services, then SATRA will be entitled to cancel or reschedule the delivery of Goods or Services at its discretion. In the event of cancellation SATRA will be entitled to retain all fees paid by the Client for Goods or Services already supplied but will refund to the Client any fees paid by the Client for Goods or Services which have not yet been supplied. The Client will not be liable for any non-refundable expenses already incurred by SATRA in relation to Goods or Services not yet supplied unless the cancellation is due to the Client's failure to comply with its obligations under the Contract.

5. LIABILITY AND INDEMNIFICATION

- 5.1 Reports are issued on the basis of information, documents and/or samples submitted to SATRA by the Client, or on behalf of the Client and are provided solely for the benefit of the Client who is responsible for acting as it sees fit on the basis of such reports and findings. Subject to clause 5.2, neither SATRA nor any of its employees, agents or subcontractors shall be liable to the Client or any third party for any actions taken or not taken on the basis of such findings and reports, nor for any incorrect results arising as a result of unclear, erroneous, incomplete, misleading or false information provided to SATRA.
- 5.2 Nothing in these terms and conditions shall limit or exclude SATRA's liability for:
 - (a) death or personal injury caused by its negligence or the negligence of its employees or agents;
 - (b) fraud or fraudulent misrepresentation;
 - (c) breach of the terms implied by Section 12 of the Sale of Goods Act 1979;
 - (d) defective products under the Consumer Protection Act 1987; or
 - (e) any other liability which cannot be limited or excluded by applicable law.
- 5.3 Subject to clause 5.2 SATRA shall not be liable to the Client whether in contract, tort (including negligence), breach of statutory duty or otherwise arising under or in connection with the Contract for loss of profits, sales, contracts, anticipated savings, loss or damage to goodwill or any indirect or consequential loss.
- 5.4 Subject to clause 5.2 SATRA's total aggregate liability to the Client, whether in contract, tort (including negligence), breach of statutory duty or otherwise arising under or in connection with the Contract shall be limited to the total amount of fees for the Services or the price of the Goods (excluding any value added tax or other sales tax or expenses) payable by the Client to SATRA under the Contract or £100,000 whichever is the lower figure.

6. MISCELLANEOUS

- 6.1 If any one or more provisions of these conditions are found to be illegal or unenforceable in any respect the validity, legality and enforceability of the remaining provisions shall not in any way be affected or impaired thereby.
- 6.2 During the course of providing the Goods or Services and for a period of one year thereafter the Client shall not directly or indirectly entice, encourage or make any offer to SATRA's employees to leave their employment with SATRA.
- 6.3 The use of SATRA's corporate name or registered marks for advertising purposes is not permitted without SATRA's prior written authorisation.
- 6.4 All reports and documentation which are supplied to the Client under the Contract remain the property of SATRA until paid in full. Under no circumstances will a Client's purchase order constitute SATRA's retention of title in accordance with this clause.
- 6.5 The Client acknowledges that in entering into this Contract it has not relied on any representation, warranty, collateral contract or other assurance (except those set out or referred to in these terms and conditions) made by or on behalf of SATRA or any other party before entering into the Contract. The Client waives all rights and remedies that, but for this clause, might otherwise be available to it in respect of any such representation, warranty, collateral contract or other assurance.
- 6.6 All provisions of the Contract that limit or exclude the liability of SATRA are intended also to be for the benefit of SATRA's holding company (called SATRA, and being a company limited by guarantee and incorporated in England and Wales with company number 00153475) and shall accordingly be enforceable by such holding company as well as or instead of by SATRA, and on the basis that any limit on the liability of SATRA shall apply to it and to such holding company in the aggregate.

7. CONFIDENTIALITY

- 7.1 Unless specifically excluded in the terms of an individual contract between SATRA and the Client, the following shall apply to all deliverables including, reports, advice, drawings, photographs, specifications, data or other forms of media.
- 7.2 Deliverables referred to in clause 7.1 shall not be disclosed to third parties or used in litigation without the consent of SATRA.
- 7.3 Where SATRA has given consent to disclosure of any service deliverables referred to in clause 7.1, the Client shall draw the attention of the third party to these terms of business and the basis on which SATRA undertakes testing, reporting and advising. The Client shall indemnify SATRA for any failure to do so.
- 7.4 The service deliverables referred to in clause 7.1 are submitted to the Client as confidential documents. Confidentiality shall continue to apply after completion of the business, but shall cease to apply to information or knowledge which has come into the public domain through no breach of this Contract by the Client.
- 7.5 The Client shall not disassemble, remove parts or carry out any form of analysis on goods or materials sold by SATRA for the purposes of reverse engineering or obtaining information on the construction, content or composition of the item without the consent of SATRA.

8. AMENDMENT

- 8.1 No amendment to this Contract shall be effective unless it is in writing, expressly stated to amend this Contract and signed by an authorised signatory of both Parties.


9. DISPUTE RESOLUTION

- 9.1 If there should be a dispute between the parties to this Agreement they undertake to act with goodwill and to use all reasonable endeavours to resolve that dispute.
- 9.2 Failure to resolve any dispute by discussions between the parties shall, in the first instance, be referred to a mediator for resolution. The parties shall attempt to agree upon the appointment of a mediator, upon receipt, by either of them, of a written notice to concur in such appointment. Should the parties fail to agree within 21 days, either party, upon giving written notice, may apply to the President or the Vice President, for the time being, of the Chartered Institute of Arbitrators, for the appointment of a mediator.
- 9.3 Should the mediation fail, in whole or in part, either party may, upon giving written notice, and within twenty-eight days thereof, apply to the President or the Vice President, for the time being, of the Chartered Institute of Arbitrators, for the appointment of a single arbitrator, for final resolution. The arbitrator shall have no connection with the mediator or the mediation proceedings, unless both parties have consented in writing. The arbitration shall be governed by both the Arbitration Act 1996 and the Controlled Cost Rules of

Nam Viet Glove Joint Stock Company
SATRA Reference: CHM0259430/1727/CL
Date: 14th July 2017

(Page 4 of 5)

Signed:

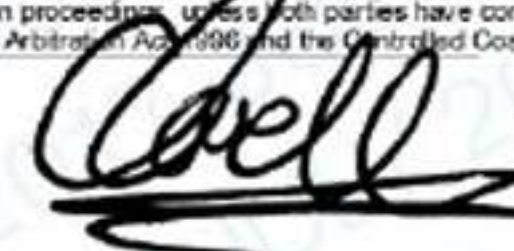

TERMS AND CONDITIONS FOR THE SALE OF GOODS AND/OR THE PROVISION OF SERVICES

- the Chartered Institute of Arbitrators (2000 Edition), or any amendments thereof, which Rules are deemed to be incorporated by reference into this clause. The seat of the arbitration shall be England and Wales. The laws of England shall govern the interpretation of this Contract. Subject to clauses 9.1, 9.2 and 9.3 any dispute arising out of or in connection with the Contract shall be subject to the exclusive jurisdiction of the courts of England. However, the Party obtaining a judgement in such courts shall be entitled to enforce it in any court it chooses.
- 10. PROVISION OF SERVICES**
- 10.1 SATRA shall provide Services using reasonable care and skill and in accordance with the Client's specific instructions and as confirmed by SATRA as part of the Contract review process.
- 10.2 Estimates for completion of the Services are made in good faith and date from receipt of a written order, payment of a proforma invoice if required, full information and samples to enable SATRA to proceed. While SATRA will make every effort to fulfil them, such estimates are subject to unforeseen events and if not achieved, cannot give rise to any claim. Time will not be of the essence in relation to the performance of the Services.
- 10.3 Results given in test reports or certificates refer only to samples submitted for analysis to SATRA. A satisfactory test report in no way implies that the product tested is approved by SATRA and no warranty is given as to the performance of the product tested.
- 10.4 SATRA may delegate all or part of the Services to a subcontractor and the Client authorises SATRA to disclose all information required to undertake the Services.
- 10.5 Where the Client requests SATRA to witness testing of other services being undertaken by a third party the Client agrees that SATRA's sole responsibility is to be present at the time of the work and to forward the results or confirm that the service has been undertaken. The Client agrees that unless otherwise agreed SATRA is not responsible for the condition or calibration of any equipment unless provided by SATRA.
- 10.6 Unless otherwise agreed in advance, test samples will be retained for 6 weeks from the date of the final report after which time they will be disposed of and SATRA shall cease to have any responsibility for such samples.
- Where the nature of the samples or the Services undertaken results in specialist disposal then SATRA reserves the right to pass the cost of such disposal onto the Client. Storage for longer periods may be possible only if agreed in advance and may incur a storage charge payable by the Client.
- Where practical and agreed in advance, samples may be returned at the Client's expense. However, samples are in most instances partially or fully destroyed as part of the work undertaken and SATRA cannot guarantee that samples will be returned in an "as new" condition.
- 10.7 Where SATRA receives documents reflecting engagements between the Client and third parties or documents belonging to third parties, such documents shall be considered as being for information only and shall not release the Client from any or all obligations to SATRA.
- 10.8 SATRA reserves the right to make changes to the Services, provided that such changes do not materially affect the nature or quality of the provision of these Services or where they are necessary in order to ensure that any applicable laws or safety requirements are complied with.
- 10.9 The Client acknowledges that SATRA by providing the Services, neither takes the place of the Client or any third party or releases them from any of their obligations.
- 11. CLIENT RESPONSIBILITIES RELATING TO THE PROVISION OF SERVICES**
- 11.1 The Client shall provide sufficient samples, information, instructions and documents as required to enable SATRA to carry out the Services in accordance with the methods, standards or other specifications as agreed.
- 11.2 Where applicable the Client shall allow access by members of SATRA staff to such premises where the Services are to be performed and provide any specialist equipment and personnel.
- 11.3 The Client shall inform SATRA in advance of any known hazards, dangers or other safety matters relating to samples submitted to SATRA or on site visits made by SATRA.
- 11.4 Where the Client fails to comply with any of its responsibilities SATRA reserves the right to suspend any Services until such time as the Client has complied and may require the Client to reimburse SATRA the amount of any additional costs arising from the suspension.
- 12. DELIVERY AND NON-DELIVERY OF GOODS**
- 12.1 Delivery dates for the supply of the Goods are approximate only and not guaranteed. Time of delivery is not of the essence of the Contract and SATRA shall not be liable for any delay in delivery of Goods.
- 12.2 Should expedited delivery be requested and agreed, SATRA shall be entitled to make additional charges to cover overtime or any other additional costs.
- 12.3 Delivery of the Goods shall take place at such location as SATRA and the Client agree. If the Client agrees to collect the Goods from SATRA's premises, then delivery will take place at those premises in which case the consignment of Goods as recorded by SATRA upon dispatch shall be evidence of the Goods received by the Client unless the Client can provide conclusive evidence to the contrary.
- 12.4 SATRA shall not be liable for the non-delivery of Goods (even if caused by SATRA) unless the Client provides written notice of non-delivery in accordance with clause 13.2. Liability for non-delivery of Goods shall in any event be limited to replacing the Goods within a reasonable time frame or the issue of a credit note to the value of the Goods not delivered.
- 12.5 Should delivery of the Goods be suspended or delayed by the Client for any reason SATRA reserves the right to charge for storage and for all expenses incurred, including loss of or wastage of resources that cannot otherwise be used. If the delay extends beyond 30 days SATRA shall be entitled to immediate payment for any Goods that are ready for delivery, and any other additional costs.
- 12.6 If for any reason the Client fails to accept delivery of any of the Goods when they are ready for delivery, or SATRA is unable to deliver the Goods on time because the Client has not provided appropriate instructions, documents, licences or authorisations then risk in the Goods shall pass to the Client, the Goods and/or Services shall be deemed to have been delivered, and SATRA may store the Goods until delivery, whereupon the Client shall be liable for all related costs and expenses (including, without limitation, storage and insurance).
- 13. RISK/TITLE OF GOODS**
- 13.1 Subject to clause 12.6 the risk in the Goods will transfer to the Client on delivery of the Goods unless SATRA and the Client have agreed that the sale of the Goods will be governed by Incoterms 2010 (or any subsequent revision thereto) in which case risk will transfer to the Client in accordance with the Incoterms mode of transport which is agreed by SATRA and the Client.
- 13.2 The Company shall not accept responsibility for loss or damage in transit unless:
- In the case of sales where delivery of Goods is made in the United Kingdom SATRA is notified by the Client within 10 days of the invoice date of non-arrival of Goods and within 3 days of the invoice date of receipt of Goods damaged in transit; or
 - In all other cases the Client notifies SATRA on the non-arrival or damage in transit within a reasonable period of time as determined by SATRA.
- 13.3 Title to the Goods shall not pass to the Client until the earlier of when: -
- SATRA receives payment in full (in cash or cleared funds) for the Goods and any other Goods that SATRA has supplied to the Client in which case title to the Goods shall pass at the time of payment of all such sums; and
 - the Client resells the Goods in accordance with clause 13.5 in which case title shall pass to the Client immediately before the time at which the resale by the Client occurs.
- 13.4 Until ownership of Goods has passed to the Client, the Client shall:
- hold the Goods as SATRA's bailee;
 - store the Goods (at no cost to SATRA) separately from all other goods belonging to the Client or any third party in such a way that they remain readily identifiable as SATRA's property (including where the Goods have been sold to a 3rd party);
 - not destroy, deface or obscure any identifying mark or packaging on or relating to the Goods; and
 - maintain the Goods in satisfactory condition and keep them insured on SATRA's behalf for their full price against all risks to the reasonable satisfaction of SATRA. The Client shall obtain an endorsement of SATRA's interest in the goods on its insurance policy. On request the Client shall allow SATRA to inspect such Goods and shall produce the policy of insurance.
- 13.5 The Client may resell the Goods before ownership has passed to it solely on condition that sale shall be effected in the ordinary course of the Client's business at full market value.
- 13.6 If before title to the Goods passes to the Client, the Client becomes subject to any of the events referred to in clause 2.6 then without limiting any other right or remedy SATRA may have:
- the Client's right to resell the Goods or use them in the ordinary course of its business ceases immediately; and
 - SATRA may at any time require the Client to deliver up all Goods in its possession that have not been resold or irrevocably incorporated into another product; and
 - if the Client fails to do so promptly SATRA may exercise its rights under clause 13.7.
- 13.7 The Client grants SATRA, its agents and employees an irrevocable licence at any time to enter any premises where the Goods are or may be stored in order to inspect them, or where the Client's right to possession has terminated, to recover them.
- 13.8 On termination of the Contract, however caused, SATRA's (but not the Client's) rights contained in this clause 13 shall remain in effect.
- 14. PATENTS**
- 14.1 SATRA gives no indemnity against any claim of infringement of Letters Patent, Registered Design, Trade Mark or Copyright by the use of or sale of any article or material supplied to the Client. If its use is impossible without infringement of Letters Patent, Registered Design, Trade Mark or Copyright published at the date of the contract, SATRA will refund to the Client the purchase price of the said article or material provided that it is returned to SATRA free of charge. The Client warrants that any design or instruction furnished or given by the Client shall not be such as will cause SATRA to infringe any Letters Patent, Registered Design, Trade Mark or Copyright in the execution of the Client's order.
- 15. WARRANTY OF GOODS**
- 15.1 SATRA warrants that on delivery and for a period of 12 months from the date of delivery or within the shelf life of the Goods (whichever is the shorter period) the Goods shall be free from defects in design, material and workmanship.
- 16. DEFECTIVE GOODS**
- 16.1 Subject to clauses 16.6 and 16.7 if:
- the Client gives notice in writing to SATRA in accordance with clause 16.3 and during the period referred to in clause 15.1 that the Goods do not comply with the warranty in that clause; and
 - SATRA is given a reasonable opportunity of examining such Goods; and
 - the Client (if asked to do so by SATRA) returns such Goods to SATRA's place of business then SATRA will, at its option, repair or replace the defective Goods or refund the price of the defective Goods in full. SATRA reserves the right to repair the Goods at the Client's premises.
- 16.2 The Client must inspect all Goods upon delivery. Failure to do so may result in further charges being applied in the event of a return.
- 16.3 If Goods are found to be faulty, defective or damaged the Client must inform SATRA in writing as soon as reasonably possible and in any event within 10 working days of the fault, damage or defect being discovered.
- 16.4 Without prejudice to clause 16.1 if no notice of rejection has been received by SATRA within 3 months of delivery, the Client shall be deemed to have accepted the Goods.
- 16.5 SATRA will pay the reasonable costs of carriage, packaging and insurance for any defective Goods which are returned by the Client provided that SATRA is liable under clause 16.1 to repair or replace the defective Goods. If SATRA determines that the Goods are not defective or if SATRA is not liable to repair or replace the Goods due to the circumstances under clauses 16.6 or 16.7 then the Client will be responsible for the payment of such costs.
- 16.6 SATRA shall not be under any liability to repair or at its option replace or pay for the repair or replacement of any Goods which are found to be defective if:
- the defect is caused or substantially caused by wear and tear, overloading, misuse, neglect, modification or attempted modification carried out by any organisation other than by SATRA or their approved agents, or use with ancillary equipment not approved in writing by SATRA, or default in proper maintenance or cleaning; or
 - the Client authorises or carries out any repair or replacement of any Goods without first affording SATRA a reasonable opportunity to replace or repair them; or
 - the Client has breached any of the terms of the Contract under which the Goods were supplied; or
 - the Goods have been manufactured to a design or specification or in compliance with other information provided by the Client and the defect has arisen as a result of that design, specification or information.
- 16.7 Where Goods or parts of Goods are not manufactured by SATRA then SATRA shall be liable for defects only to the extent that SATRA obtains redress from the manufacturer or supplier thereof provided that:
- SATRA shall not be obliged to take any step to attempt to obtain such redress except at the request and expense of the Client and upon provision by the Client of a full indemnity as to costs for which SATRA may thereby become liable;
 - nothing in this condition 16.7 shall have effect as to impose upon SATRA any additional liability or obligations other than those referred to in condition 16.1.
- 16.8 Except as provided in clause 16.1 SATRA shall have no liability to the Client arising from any failure of the Goods to comply with the warranty in clause 15.1.

Terms and conditions – December 2016

Nam Viet Glove Joint Stock Company
SATRA Reference: CHM0259430/1727/CL
Date: 14th July 2017

Signed:





SATRA Technology Centre Ltd
Wyndham Way, Telford Way, Kettering,
Northamptonshire, NN16 8SD United Kingdom
Tel: +44 (0) 1536 410000
Fax +44 (0) 1536 410626
email: info@satra.com
www.satra.com



Customer details: Nam Viet Glove Joint Stock Company
23a Nguyen Huu Canh St
Ward 22
Binh Thanh District
HCMC
Vietnam

SATRA reference: SPC0257773 /1721

Your reference:

Date of report: 3 June 2017

Samples received: 23 May 2017

For the attention of: Ooi Loon Seng

Date(s) work carried out: 31 May to 3 June 2017

TECHNICAL REPORT

Subject: Testing of gloves identified as NPF-FT30-24LB in accordance with EN 420: 2003 + A1: 2009 clauses 5.1 Sizing and 5.2 Dexterity and EN 374-2: 2014

Conditions of Issue:

This report may be forwarded to other parties provided that it is not changed in any way. It must not be published, for example by including it in advertisements, without the prior, written permission of SATRA.

Results given in this report refer only to the samples submitted for analysis and tested by SATRA. Comments are for guidance only.

Tests marked \neq fall outside the UKAS Accreditation Schedule for SATRA. All interpretations of results of such tests and the comments based upon them are outside the scope of UKAS accreditation and are based on current SATRA knowledge.

A satisfactory test report in no way implies that the product tested is approved by SATRA and no warranty is given as to the performance of the product tested. SATRA shall not be liable for any subsequent loss or damage incurred by the client as a result of information supplied in the report.

The uncertainty of the results (UoM) in this report is based on a standard uncertainty multiplied by a coverage factor $k=2$, which provides for a confidence level of approximately 95%.

Report signed by: Adam Mortiboys
Position: PPE Technologist
Department: Safety Product Testing

(Page 1 of 5)

Work Requested

Samples of gloves, see Table 1, were received by SATRA, for testing in accordance with EN 420:2003+A1:2009 clauses 5.1 length and fit and 5.2 dexterity and EN 374-2:2014.

Table 1 – Samples Received

Sample description as stated by the client	Sizes submitted for testing	Colour of samples submitted	Approximate weight of one glove
NPF-FT30-24LB	S – XL	Blue	Size: XL Weight: 3.8g



Conclusion

Standard	Clause / Property	Result
EN 420: 2003 + A1: 2009	5.1 Length and fit	See note ■
	5.2 Dexterity	Level 5
EN 374-2: 2014	7.2 Air leak	Pass
	7.3 Water leak	Pass

Note ■ – Where gloves do not meet the minimum length requirements specified in Table 1 of EN 420:2003 + A1:2009, the standard therefore requires that the manufacturer shall clearly state in the user instructions the intended application of the gloves and the reason why the gloves do not conform to the minimum length requirements.

Testing

Samples for testing were conditioned for at least 24 hours in a conditioned environment maintained at $(23 \pm 2) ^\circ\text{C}$ and $(50 \pm 5) \%$ relative humidity. Testing was carried out within the same environment.

Requirements

Table 2 – Requirements for EN 420:2003 + A1:2009 Clause 5 Size and Dexterity

Glove size	6	7	8	9	10	11
Minimum length / mm	220	230	240	250	260	270

Performance level	1	2	3	4	5
Diameter of dexterity pin /mm	11.0	9.5	8.0	6.5	5.0

Table 3 - Requirements for EN 374-2: 2014

7.2 Air leak test	No leak to be detected
7.3 Water leak test	No leak to be detected

Test Results

Table 4 - EN 420:2003 + A1:2009 Test Results for gloves identified as NPF-FT30-24LB

Clause / Test	Test Results		UoM	Result
5.1 Glove length, comfort and fit	Size	Length /mm		
	S	Left 245 Right 247 Comments on fit: Satisfactory		
	M	255 254 Comments on fit: Satisfactory	± 0.3 mm	See note
	L	254 253 Comments on fit: Satisfactory		
5.2 Dexterity	XL	245 244 Comments on fit: Satisfactory		
	Size	Minimum pin diameter / mm		
	S	5.0	N/A	Level 5
	M	5.0		
	L	5.0		
	XL	5.0		

Table 5 - EN 374-2:2014 Test Results of gloves identified as NPF-FT30-24LB

Clause / Test	Test Results		UoM	Result
7.2 Air leak test	Total Air Pressure Used	2.39 kPa		
	Sample size	Leaks		
	S	No leaks detected	± 2.8 mmH ₂ O	Pass
	M	No leaks detected		
7.3 Water leak test	L	No leaks detected		
	XL	No leaks detected		
	Sample size	Leaks		
	S	No leaks detected	N/A	Pass
	M	No leaks detected		
	L	No leaks detected		
	XL	No leaks detected		

TERMS AND CONDITIONS OF BUSINESS

1. **GENERAL**
Work done or services undertaken are subject to the terms and conditions detailed below and all other conditions, warranties and representations, expressed or implied are hereby excluded.
2. **PRICES**
Prices are based on current material and production costs, exchange rates, duty and freight and are subject to change without notice.
3. **DELIVERY ESTIMATES**
Delivery estimates are made in good faith and date from receipt of a written order and full information to enable us to proceed. While SATRA or its subsidiaries (hereafter referred to as "SATRA") make every effort to fulfil them, such estimates are subject to unforeseen events and if not maintained, cannot give rise to any claim. Offers "ex stock" are subject to prior sale.
4. **CANCELLATION AND RETURNS**
Cancellation of orders for goods, services, training or consultancy is only acceptable by prior agreement of SATRA and a charge will normally be made.
5. **CLAIMS**
Claims for errors, shortages etc should be notified within 10 days of date of receipt. In the event of goods damaged in transit, packing materials should be retained for examination; otherwise no liability can be accepted.
6. **PAYMENT TERMS**
Payment terms are net 21 days from date of invoice. Failure to comply with the terms of payment may result in delayed delivery of goods and services and a review of the Customer's credit account. Should the customer become subject to an administration order, or becomes bankrupt or goes into liquidation, SATRA has a right to cancel any contract and discontinue any work. SATRA reserves the right to adjust US Dollar and Euro sales price where customer exceeds credit terms and where the exchange rate has moved more than 10% since invoicing.
7. **RETENTION OF TITLE**
All goods remain the property of SATRA until paid in full. Under no circumstances will a customer's purchase order override SATRA's Retention of Title clause. In the case of software, the ownership of the software remains with SATRA. Payment of invoices in full will entitle the customer to use the software under licence until (a) they cease to be a member of SATRA or (b) they cease trading. In both instances, the licence shall then revert to SATRA.
8. **GUARANTEE**
All goods manufactured by SATRA are guaranteed both as regards material and workmanship. Any part returned carriage paid, within twelve months from date of supply and found defective, will be repaired or replaced at SATRA's option free of charge. SATRA admits no liability for loss, damage or delay consequent on any defect in any goods supplied by SATRA.
9. **TEST REPORTS**
Results given in test reports refer only to samples submitted for analysis and tested by SATRA. A satisfactory test report in no way implies that the product tested is approved by SATRA and no warranty is given as to the performance of the product tested. SATRA shall not be liable for any subsequent loss or damage incurred by the Customer as a result of information supplied in a test report.
10. **TEST SAMPLES**
Unless otherwise agreed in advance, test samples will be disposed of 6 weeks after the date of the final report. If required, samples can be returned at the Customer's expense.
11. **RESPONSIBILITY**
Every effort is made to ensure accuracy in description, drawings and other information in correspondence, catalogues, etc but no warranty is given in this respect and SATRA shall not be liable for any error therein. SATRA carries out all tests and/or advises only on the basis that the same are carried out, made or given without any responsibility whether for negligence or otherwise. SATRA and its servants or agents will not be liable for any damage or loss direct or indirect of whatsoever kind, whether or not the same results directly or indirectly from negligence on the part of SATRA or its servants or agents.
12. **CONFIDENTIALITY**
Unless specifically excluded in the terms of an individual contract between SATRA and its Customer, the following shall apply to all reports, advice, drawings, photographs, specifications or data:
 - i. The above shall not be disclosed to third parties or used in litigation without the consent of SATRA.
 - ii. Where SATRA has given consent to disclosure, the Customer shall draw the attention of the third party to these terms of business and the basis on which SATRA undertakes test, reporting and advising. The Customer shall indemnify SATRA for any failure to do so.
 - iii. The above items are submitted to the Customer as confidential documents. Confidentiality shall continue to apply after completion of the business, but shall cease to apply to information or knowledge which may come into the public domain.
13. **CONSTRUCTION AND ARBITRATION**
The laws of England shall govern all contracts and the parties submit to exclusive jurisdiction of the courts of England, unless otherwise agreed.

Issue Date: 1st October 2009